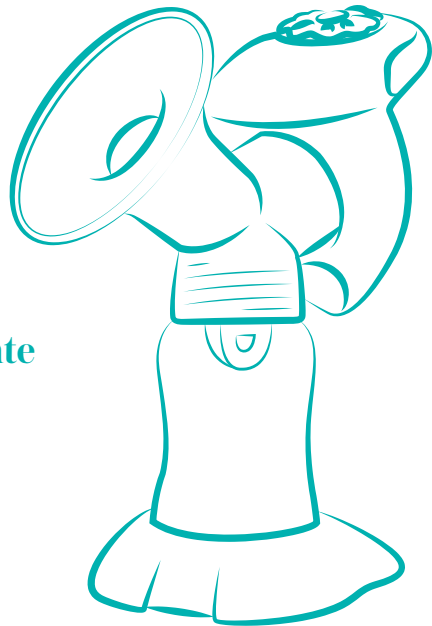
# Advanced Single Electric Breast Pump

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For video instructions about how to use and care for your pump,  
visit: [www.evenflofeeding.com](http://www.evenflofeeding.com)

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*Thank you for choosing Evenflo Feeding to support you on your breastfeeding journey.*

***If you have questions about the performance or operation of this pump, please contact our Parent's Ally Resource Center.***

*For more information about breastfeeding or pumping, contact a lactation expert, your physician, or visit one of the websites listed below.*

*USA/Canada: 1-855-EFI-BABY (334-2229) or visit us online at **[www.evenflofeeding.com](http://www.evenflofeeding.com)**.*

### ***International Lactation Consultant Association***

*Lactation consultants with the IBCLC credential specialize in the clinical management of breastfeeding. There may be a fee to use lactation consultant services. To locate one near you, call: (919) 861-5577. Website: **[www.ilca.org](http://www.ilca.org)***

### ***La Leche League***

*Provides free mother-to-mother breastfeeding support.*

*Phone: 1-800-LALECHE (US), or (847) 519-7730. Website: **[www.llli.org](http://www.llli.org)***

### ***Office on Women's Health***

*Provides trained breastfeeding peer counselors for free telephone help with common issues. Phone: 1-800-994-9662. Website: **[www.womenshealth.gov](http://www.womenshealth.gov)***

### ***WIC (Women, Infant and Children Program)***

*Provides breastfeeding help to mothers and children enrolled in the WIC program. Website: **[www.fns.usda.gov](http://www.fns.usda.gov)***

### ***Babies in the Workplace***

*Website: **[www.babiesatwork.org](http://www.babiesatwork.org)***

### ***Breastfeeding and Working***

*Website: **[www.workandpump.com](http://www.workandpump.com)***

### ***Kelly Mom Breastfeeding and Parenting***

*Information on multiple topics. Website: **[www.kellymom.com](http://www.kellymom.com)***

### ***Low Milk Supply Information and Support***

*For mothers experiencing low milk supply.*

*Website: **[www.lowmilksupply.org](http://www.lowmilksupply.org)***

### ***Breastfeeding After a Reduction***

*Information for women who are breastfeeding after a breast reduction.*

*Website: **[www.bfar.org](http://www.bfar.org)***

**Please Note:** This pump is designed and intended for occasional use only, meaning **no more than 2 pumping sessions per day**. Refer to the chart on the back of these instructions for examples of appropriate situations for occasional use. If this pump is used more than twice daily, your pump's life expectancy may diminish earlier. Also, using this pump more than twice per day may impact your pumping success and/or your health. Moms who work full-time, or pump more than occasionally, should use the Evenflo Advanced Double Electric Breast Pump, which has been expertly developed for frequent pumping.

**Indications for Use:** The Evenflo Advanced Single Electric Breast Pump is an electrically-powered suction device intended to express and collect milk from a lactating woman's breasts. This pump is intended for a single user.

Please note that this pump should not be serviced or disassembled by the consumer. Once the package has been opened, this product may not be returned to the store. If you have any issues or concerns with the product, please contact our Parent's Ally Resource Center.

## Table of Contents

Important Safeguards and Warnings .....	2
Identification of Parts .....	4
Before First Use.....	4
Assemble Your Breast Pump.....	6
Using Batteries .....	7
Using the AC Adapter.....	7
Check the Fit of the Flange .....	8
Using Your Breast Pump .....	9
After Every Use .....	11
Storing Your Breast Milk .....	11
Using Your Breast Milk .....	12
Troubleshooting .....	13
Limited Warranty .....	15

# **▲ IMPORTANT SAFEGUARDS AND WARNINGS**

**Read all instructions before using and save for future use.**

*When using this pump and other electrical products, basic safety precautions should always be maintained, including the following:*

## **DANGER: To reduce the risk of electrocution**

- **ALWAYS** unplug this product immediately after using.
- **DO NOT** use while bathing.
- **DO NOT** place or store this product where it can fall or be pulled into a tub or sink.
- **DO NOT** place in or drop into water or other liquid.
- **DO NOT** reach for an electrical product that has fallen into water. Unplug the product immediately.
- **NEVER** immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven.

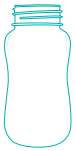
## **To reduce the risk of burns, electrocution, fire or injury to persons:**

- Use this product only for its intended use as described in these instructions.
- Breast pumps are personal care items, for one user only.
- This product should never be left unattended when plugged in.
- **NEVER** use while sleepy or drowsy.
- Close supervision is necessary when this product is used near children.
- **DO NOT** allow small children or pets to play with the pump unit, flange, AC adapter, etc. These parts may pose a choking or strangulation hazard.
- Pump contains small parts which could cause a child to choke. Keep pump parts and accessories out of children's reach.
- **DO NOT** use attachments not recommended by the manufacturer.
- **DO NOT** use this product if it is broken or any parts are missing.
- **NEVER** drop or insert any object into any opening of the pump unit.
- **DO NOT** use flange insert if your nipple is larger than the flange insert's center opening. See page 8 for proper flange fit.
- If you are experiencing engorgement, prevent possible injury by discontinuing use of this pump and consulting a medical provider.
- **NEVER** operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, dropped into water, or if any parts are broken or missing.
- Keep the cord away from heated surfaces.
- **DO NOT** use outdoors with the AC adapter or operate where aerosol spray products are being used, or oxygen is being administered.


## **⚠ IMPORTANT SAFEGUARDS AND WARNINGS**

- Operate pump using **ONLY** 3 “AA” name-brand batteries (not included) or with the Evenflo AC adapter provided, and plug the adapter into a standard wall outlet.
- Operate the AC adapter on 100 – 240 volts 50/60 Hz AC only. Connect only to household outlet power supplies of this description. For use in countries outside North America, a plug converter (not included) may be required.
- Protect power cord and AC adapter cord. Route cords so that they are not walked on or pinched by items placed upon them.
- **DO NOT**, under any circumstances, use this AC adapter with any other equipment. Likewise, do not use the adapter from any other product on this breast pump.
- Power adapter must be removed for battery operation. In case of a power outage, batteries will not act as a backup if the adapter is plugged into the pump.
- **DO NOT** mix old and new batteries. **DO NOT** mix alkaline, standard (carbon-zinc), or rechargeable batteries; leakage, explosion, and/or fire could occur.
- Remove batteries before storing unit for a prolonged period of time.
- **DO NOT** heat breast milk in the microwave. Microwaving can destroy nutrients in breast milk and can heat liquids unevenly resulting in burns or scalds. See page 11 for proper use of breast milk.
- **DO NOT** re-freeze breast milk. After feeding, any unused breast milk left in the bottle can be refrigerated and used within 4 to 6 hours. See page 12 for proper milk storage.
- **ALWAYS** test temperature of liquid before feeding.
- If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), pumping breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.
- **Save These Instructions for Future Use.**


## Identification of Parts


Bottle

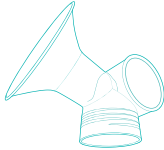
Flange  
(30.5mm)

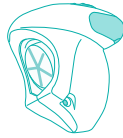Single Pump Unit

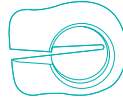Stand

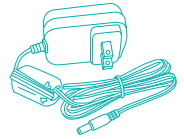AC Adapter

Valve

Membrane

Sealing Disc

Bottle Cap Ring

Diaphragm


\*28.0mm Flange Insert

\* Additional sizes are available separately. See page 8.

## Before First Use

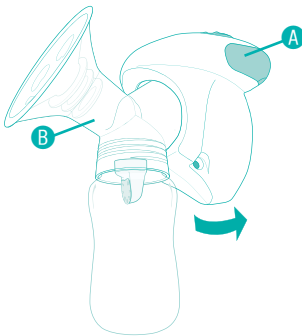
**IMPORTANT:** Before using your breast pump, disassemble and sanitize thoroughly. Always wash hands well with soap and water before handling your pump.

1


Remove the pump unit **A** from flange assembly **B** by pulling up on pump unit, as shown.

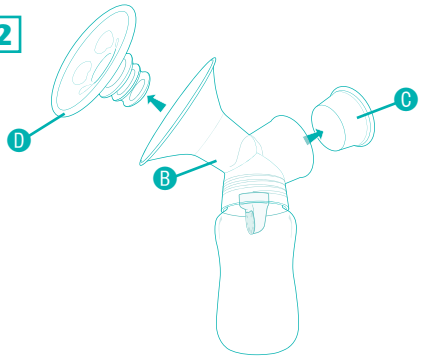2


Remove diaphragm **C** and flange insert **D** from flange assembly **B**.

**3**

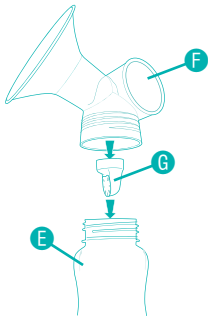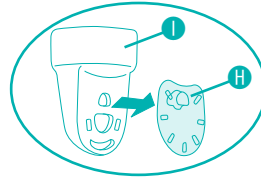Unscrew bottle **E** and twist valve **G** to remove from assembled flange unit **F**. Remove membrane **H** from valve **I**.


**To Sanitize Parts on the Stovetop:**

Place disassembled pump parts (except the pump unit and AC adapter) in a pan of boiling water for 5 minutes. **Stir frequently. DO NOT** allow parts to rest on bottom of pan as they may melt or become damaged.

**To Sanitize Parts in the Microwave:**

These pump parts may be used with a microwave sanitization device (except the pump unit and AC adapter). Please follow device manufacturer's instructions.

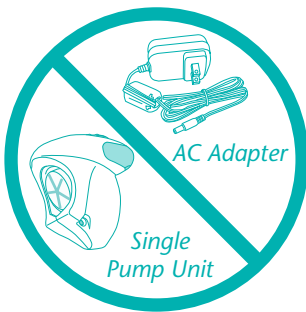**DO NOT BOIL OR WASH**

**⚠ WARNING**

**NEVER** immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven. Damage to parts and oven may occur.

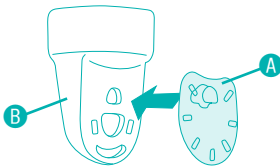
**Before EVERY Use**

- Wash your hands with soap and water before handling your pump.
- Ensure parts are clean and dry. Refer to cleaning instructions on page 11.
- Use care when handling the valve/membrane assembly to avoid damage.
- Visually inspect the individual components for cracks, chips, tears, discoloration or deterioration. In the event that damage to the device is observed, please discontinue use until the damaged parts have been replaced.

## Assemble Your Breast Pump

---


1

Attach the membrane **A** to the valve **B**.


**Note:** Make sure the membrane is attached and is lying flat against the valve, or the pump will lose suction and will not work properly. Also, make sure the membrane is not torn, as this also will cause decreased suction and the pump to not work properly.

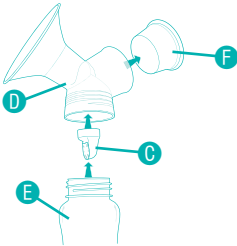2


Push valve assembly **C** onto the bottom of the flange **D**. Screw bottle **E** into bottom of flange. Install diaphragm **F** into flange assembly **D**.

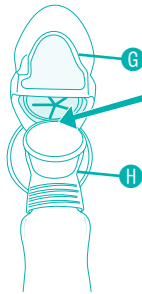3

**Align** the pump unit **G** to the flange assembly **H**, as shown.

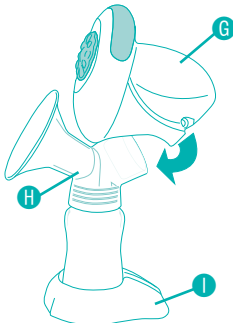
*Back View*


In a downward motion, **snap** the bottom of the pump unit **G** to the flange assembly **H**. You will feel a “click” when the two are secured tightly. Place the bottle into the stand **I**, as shown.

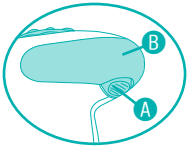*Side View*

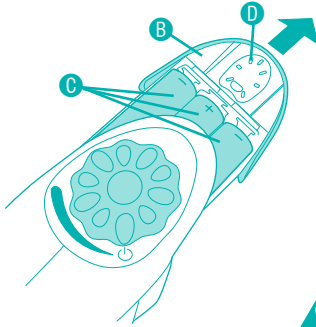If you plan to use batteries, push battery drawer release button **A**, slide battery drawer **B** out and load three alkaline “AA” batteries **C** (not included), as shown.

For best performance, brand name, non-rechargeable batteries are recommended. Generic batteries may impact pump performance.

Slide the drawer **B** closed.


**Note:** Spare membrane **D** is stored in battery drawer, as shown.

**Note:** Power adapter must be removed for battery operation. In case of a power outage, batteries will not act as a backup if the adapter is plugged into the pump.

## ⚠ WARNING

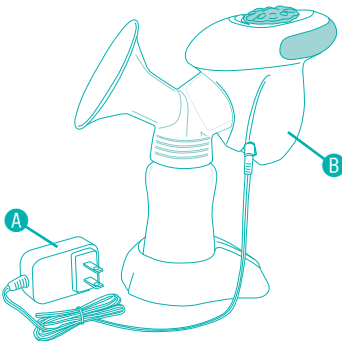- **DO NOT** mix old and new batteries.
- **DO NOT** mix battery types; leakage, explosion, and/or fire could occur.
- Remove batteries before putting product into storage.

## Using the AC Adapter

Before using the AC adapter **A**, remove batteries from pump unit.

Plug AC adapter into the pump unit **B**, then plug the adapter into 100 – 240V 50/60 Hz AC current only (standard household outlet). For use in countries outside of North America, a plug converter (not included) may be required.

Unplug the adapter after each use.

## ⚠ WARNING

- **DO NOT**, under any circumstances, use this AC adapter with any other equipment. Likewise, do not use the adapter from any other product on this breast pump. If you lose the AC adapter, contact Evenflo for a replacement.
- **ALWAYS** remove plug from product and wall by pulling on the plug/housing.
- **DO NOT** pull on the cord, as this may damage the cord.
- **NEVER** operate this product if it has a damaged cord or plug.

## Check the Fit of the Flange

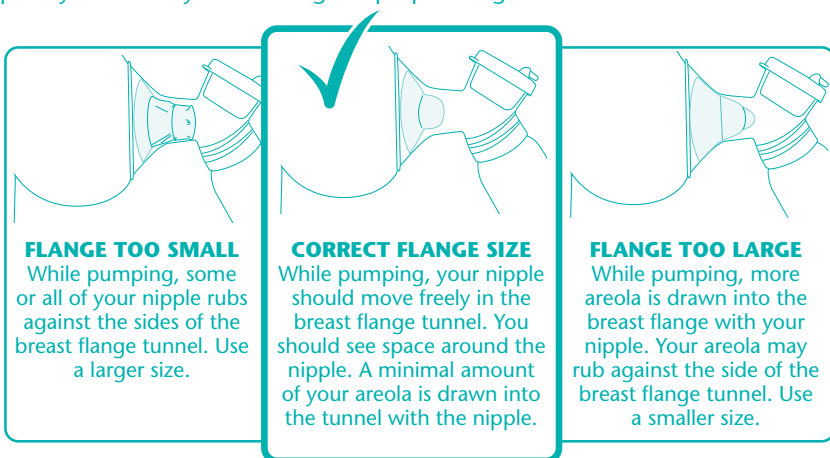
The Advanced Single Electric Pump works with the Evenflo AdvancedFit™ Flange System. The Advanced Single Electric Pump comes with the Large 30.5mm base flange size and the Medium 28.0mm silicone flange insert. Additional sizes are available separately for purchase, allowing you to find your best fit.

### The AdvancedFit™ Flange System available sizes include:

- X-Small = 21.0mm (insert)
- Small = 24.5mm (insert)
- Medium = 28.0mm (insert)
- Large = 30.5mm (no insert)
- X-Large = 33.0mm (insert)
- XX-Large = 36.0mm (no insert)

### Correct Flange Size Vs. Too Small or Too Large


A proper flange size is needed for comfort and to maintain your milk supply. As your flange size can change several times throughout your journey, recheck your flange fit frequently to ensure you are using the proper flange size.

### IMPORTANT:

- **DO NOT** use the flange insert if your nipple is larger than the center opening.
- Your breast may change over time. Should this occur, choose a different size for maximum comfort.
- **Included flange sizes may not fit all breast and nipple sizes, but additional sizes can be purchased separately.** If you are having issues after trying all included sizes, please contact our Parent's Ally Resource Center for assistance.

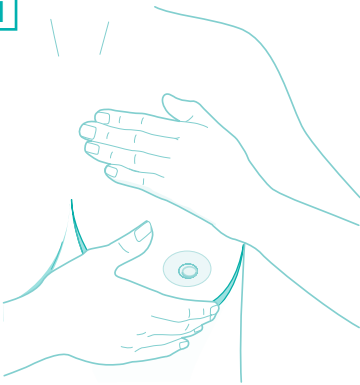1


*To maximize your pumping results, stimulate your let-down reflex by:*


- Relaxing and thinking about your baby.
- Gently massaging your breasts, starting nearest your ribs and working down toward the nipple.
- Applying warm, moist compresses; continue until you can hand-express a few drops of milk.

Now you should be ready to use your pump.

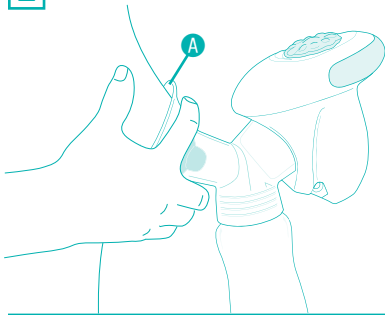2

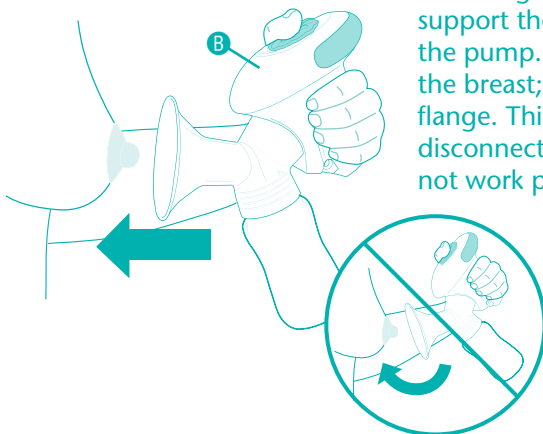For best results, hold the pump by the flange **A**. Place the flange **A** gently against your breast with your nipple centered in the flange tunnel. Press directly against your breast to create a good seal.


If holding by the pump unit **B**, do not support the weight of your breast with the pump. Press the flange directly against the breast; do not scoop breast into the flange. This may cause the pump unit to disconnect from the flange assembly and not work properly.

**Note:** Pressing too hard can restrict milk flow in your breast and may break the seal between the pump unit and the flange.


## Using Your Breast Pump

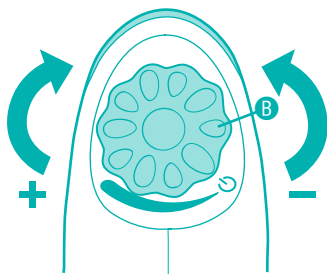3


Turn on the pump by turning the control dial **B** clockwise to begin pumping. To increase suction, gradually turn the control dial clockwise to reach the level that is most comfortable for you. To reduce suction, turn the control dial **B** counterclockwise.

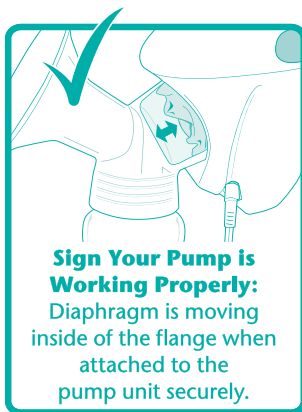**IMPORTANT:** If you feel pain in your breast or nipple, turn the pump off (see next page) or slide a finger between the flange and your breast to immediately break the suction. If you feel discomfort at every suction level, turn the pump off and contact your healthcare provider for assistance.


4

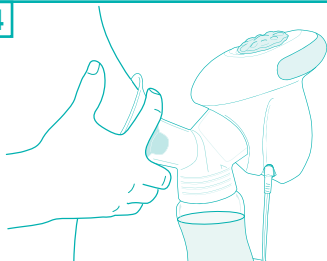Continue pumping until milk is no longer flowing or bottle is full. Do not fill bottle above the 5 oz/150 mL line.

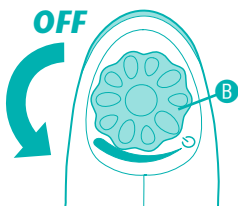
5

After breast milk has been expressed, turn control dial **B** counterclockwise until pump unit is in the OFF position. If necessary, slide a finger between the flange and your breast to break any remaining suction, then remove the flange.

Massage your other breast to stimulate a let-down, as before, and repeat steps 2 through 4. If desired, massage the first breast again and repeat on both sides to maximize expression.

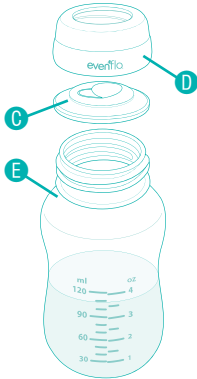6

Carefully unscrew bottle from pump. Place the sealing disc **C** into the cap ring **D** and screw securely onto the bottle **E**.

Your breast milk is now ready for storage.

**Note:** Information on storing breast milk can be found on page 11.

### After Every Use

## ⚠ WARNING

**NEVER** immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven. Damage to parts and oven may occur.

**To ensure that your breast pump performs well, follow these steps after every use:**

- Completely disassemble the flange assembly, see page 4.
- Wipe the pump unit with a clean, damp cloth.
- Rinse the flange assembly (**all parts EXCEPT** the pump unit and AC adapter) in cool water to remove milk residue.
- Hand wash with warm, soapy water.
- Rinse thoroughly and air dry before assembly. You may also wash your flange assembly (after rinsing) on the top rack of your dishwasher.
- Use care when handling the valve/membrane assembly during cleaning to avoid damage. Visually inspect the individual components for cracks, chips, tears, discoloration or deterioration. In the event that damage to the device is observed, please discontinue use until the damaged parts have been replaced.

### Storing Your Breast Milk

1. After sealing your bottle tightly with the sealing disc and cap ring, label the bottle with the date and time the milk was expressed.
2. Use the chart on the following page as a guide for storing your breast milk for future use.

**Note:** In hot weather, chill milk immediately to prevent spoilage.

## Storing Your Breast Milk

<b>Breast Milk Storage Guidelines*</b>				
	<b>Room Temperature</b>	<b>Refrigerator</b>	<b>Self-contained Refrigerator Freezer</b>	<b>Deep Freezer</b>
<b>Freshly Expressed Breastmilk</b>	4 hours at 73 – 77°F (23 – 25°C)	5 – 7 days at 32 – 39°F (0 – 4°C)	3 – 4 mos. at 0°F (-19°C)	6 – 12 mos. at 0°F (-19°C)
<b>Thawed Breastmilk (previously frozen)</b>	<b>DO NOT</b> store	24 hours	<b>NEVER</b> re-freeze thawed milk	<b>NEVER</b> re-freeze thawed milk

**Note:** Breast milk should be stored in the main refrigerator or freezer compartment, away from the door, to avoid changes in temperature.

## Using Your Breast Milk

1. Thaw slowly in the refrigerator. Avoid letting milk sit out at room temperature to thaw. (For quicker thawing, hold container under running water – start with cool water and gradually increase the temperature.)
2. Just before feeding, warm the milk by:
  - Using a bottle warmer
  - Holding the sealed bottle under warm, running water, or
  - Heating water in a cup or other small container, then placing sealed bottle in the water to warm.
3. The milk fat may separate during thawing. If so, gently swirl milk to mix before offering it to baby.
4. If feeding from the collection bottle, **you must use Evenflo Balance + Standard Nipples**. Other brands or types of nipples will not work effectively. When using the Evenflo Balance + Standard Nipples for feeding, squeeze nipple base and insert through the **TOP** of the cap ring. Once lip is inside cap ring, release the nipple and lightly push the inside lip towards the edge. Hand-tighten cap ring with care to not over or under tighten. *Evenflo Balance + Standard Nipples sold separately.*
5. Always test the temperature of the milk before feeding.

### **IMPORTANT!**

- **NEVER microwave human milk or heat it directly on the stove.**
- Previously frozen milk may be kept in the refrigerator for up to 24 hours after it has finished thawing. **DO NOT re-freeze.**
- **If baby does not finish milk at one feeding**, it may be refrigerated and offered at the next feeding before it is discarded (4 – 6 hours).
- **DO NOT** pour freshly pumped breast milk onto frozen breast milk because thawing can occur.

\* Human Milk Banking Association of North America. Best practice for expressing, storing, and handling human milk in hospitals, homes, and child care settings. Raleigh, NC: 3rd Edition, 2011.

<b>Problem: LOW OR NO SUCTION</b>	
<b>Possible Cause</b>	<b>Action</b>
Flange not assembled to pump properly.	Check assembly of flange to pump unit. All parts must be assembled properly and fit together snugly, without gaps, for the pump to provide maximum comfortable suction. Confirm you heard and felt a click while assembling (p. 6).
	When pump unit is connected properly, diaphragm should move as referenced on page 10.
	If the diaphragm is not moving when the pump is on, disconnect and reconnect the flange assembly to the pump unit (p. 6).
Control Dial set too low.	Turn the control dial clockwise for stronger suction (p. 10).
Valve not assembled to flange.	Mount the valve firmly to flange (p. 6).
Membrane worn, torn or not assembled to valve.	Replace membrane or ensure that it is connected properly to the valve (p. 6). To maintain optimal performance with daily use, it is recommended that membranes be replaced every 3 months. <b>Note:</b> Spare membrane is included in the battery door tray.
Dried milk in pump parts.	Follow cleaning instructions. This pump can be easily disassembled and washed (pp. 4 and 11).
Membrane not sealing.	Make sure valve and membrane are clean, with no debris on the surface of the seal. Make sure membrane lies flat against the surface of the valve (p. 6).
Not a good seal on your breast.	Reposition the flange to get a better seal. If the breast becomes wet with milk, you may need to dry skin and flange and start again (p. 9).
Low batteries.	Make sure fresh, brand name batteries are installed correctly (p. 7).
<b>Problem: DISCOMFORT WHEN USING PUMP</b>	
<b>Possible Cause</b>	<b>Action</b>
Suction is too high.	Turn the control dial counterclockwise to reduce the suction (p. 10).
Improper flange fit.	Reference flange fit guide on page 8. <b>Note:</b> Pump can be used with or without insert.

## Troubleshooting

---

### **Problem: PUMP WON'T TURN ON**

<b>Possible Cause</b>	<b>Action</b>
No batteries or low batteries.	Make sure fresh, brand-name batteries are installed correctly (p. 7).
AC adapter cord is still plugged into pump.	If using batteries, adapter cord must be removed from pump for batteries to work.
AC adapter is not plugged into a functioning wall outlet.	Plug AC adapter into known functioning wall outlet. Test outlet with other electrical devices.
	Try operating pump with fresh, brand-name batteries.

### **Problem: MILK WON'T FLOW INTO THE BOTTLE**

<b>Possible Cause</b>	<b>Action</b>
The valve is not opening.	Clean the valve thoroughly (p. 11) and be sure it opens.

**Still having problems?** Call our Parent's Ally Resource Center for help (inside front cover).

Remember, this is a personal care item for one user only. For hygienic reasons, it **CANNOT BE RETURNED TO THE STORE ONCE PACKAGE HAS BEEN OPENED.**

If you have any questions, comments, or concerns about our Evenflo breast pumps, please contact our Parent's Ally Resource Center.

***This product is designed for use by one person.*** Use of this product by more than one person will void the warranty.

*For a period of 90 days from the original purchase of this Product, Evenflo Feeding, Inc. ("Evenflo") warrants to the original end user ("Purchaser") this Product (including any accessories) against defects in material or workmanship. Evenflo's sole obligation under this express limited warranty shall be, at Evenflo's option, to repair or replace any Product that is determined to be defective by Evenflo and determined to be covered by this warranty.*

*Repair or replacement as provided under this warranty is the exclusive remedy of the Purchaser. **Proof of purchase in the form of a receipted invoice or bill of sale evidencing that the Product is within the warranty period must be presented to obtain warranty service.** This express limited warranty is extended by Evenflo ONLY to the original Purchaser of the Product and is not assignable or transferable to subsequent purchasers or end users of the Product. For warranty service, contact Evenflo's Parent's Ally Resource Center at 1-855-EFI-BABY (334-2229) or [www.evenflofeeding.com](http://www.evenflofeeding.com).*

*EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THIS PRODUCT IS HEREBY DISCLAIMED. EVENFLO SHALL NOT BE LIABLE FOR ANY INCIDENTAL, CONSEQUENTIAL, SPECIAL OR PUNITIVE DAMAGES OF ANY KIND FOR BREACH OF THE EXPRESS LIMITED WARRANTY ON THIS PRODUCT, OR ANY WARRANTY IMPLIED BY OPERATION OF LAW. OTHER THAN THE EXPRESS LIMITED WARRANTY SET FORTH ABOVE, THERE ARE NO OTHER WARRANTIES THAT ACCOMPANY THIS PRODUCT AND ANY ORAL, WRITTEN OR ANY OTHER EXPRESS REPRESENTATION OF ANY KIND IS HEREBY DISCLAIMED.*

**Parent's Ally**

USA/Canada: 1-855-EFI-BABY (334-2229); [www.evenflofeeding.com](http://www.evenflofeeding.com)  
Manufactured and Distributed by Evenflo Feeding, Inc.

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Fabricado y distribuido por Evenflo Feeding, Inc.