

Fitting & Care Instruction



Model: DCT-1686

Instructions on putting on brace:



Step 1: Open the brace and lay flat on a table. Detach side wings from right and left to find your waist circumference and size. Find the opening and insert cold pack if desired.



Step 2: Wrap the left front closure of the belt over the right and secure low on the abdomen.



Step 3: If setting range of motion for flexion extension, If not jump to step 5: Find the button located above the Trend logo and press in.



Step 4: While pressing the button rotate the cover and place the pin in the desired flexion and extension slots



Step 5: Place the hip bar on the side of the affected hip and line of the condyle pad on the patients trochanter. Once lined up, attach the hip bar to the side of the belt.



Step 6: Locate the telescoping bar and adjust to the height of the patient so it lines up comfortably on the patients thigh.



Step 7: Wrap the thigh cuff around the patients thigh. Extension panel included if needed.



Step 8: Locate the rubber pull tabs and tighten to desired support.



Step 9: If setting abduction: Locate the Trend key and rotate until desired abduction is reached.

Care Instructions: Hand wash with a light detergent. **Warranty:** 6 month warranty from any defects.

| Size 0 | Size 1 | Size 2 | Size 3 | Size 4 | Size 5 | Size 6 | Size 7 |
|-----------|-----------|-----------|-----------|-----------|-----------|-----------|-----------|
| 22" - 32" | 31" - 37" | 36" - 40" | 39" - 43" | 42" - 46" | 45" - 49" | 48" - 52" | 51" - 55" |