

FOLDABLE SOLID SEAT INSERT

CSH1103

WHAT'S INCLUDED


- 1x Foldable Solid Seat Insert



To see all FAQs in one place, visit vhealth.link/t0n

WHAT IS THE FOLDABLE SOLID SEAT INSERT?

The Foldable Solid Seat Insert prevents the “hammock effect” - the uncomfortable sagging that occurs in wheelchair seats after extended use. By providing a firm, level foundation beneath your cushion, this insert helps prevent potential pressure points and improves posture, enhancing both comfort and support.

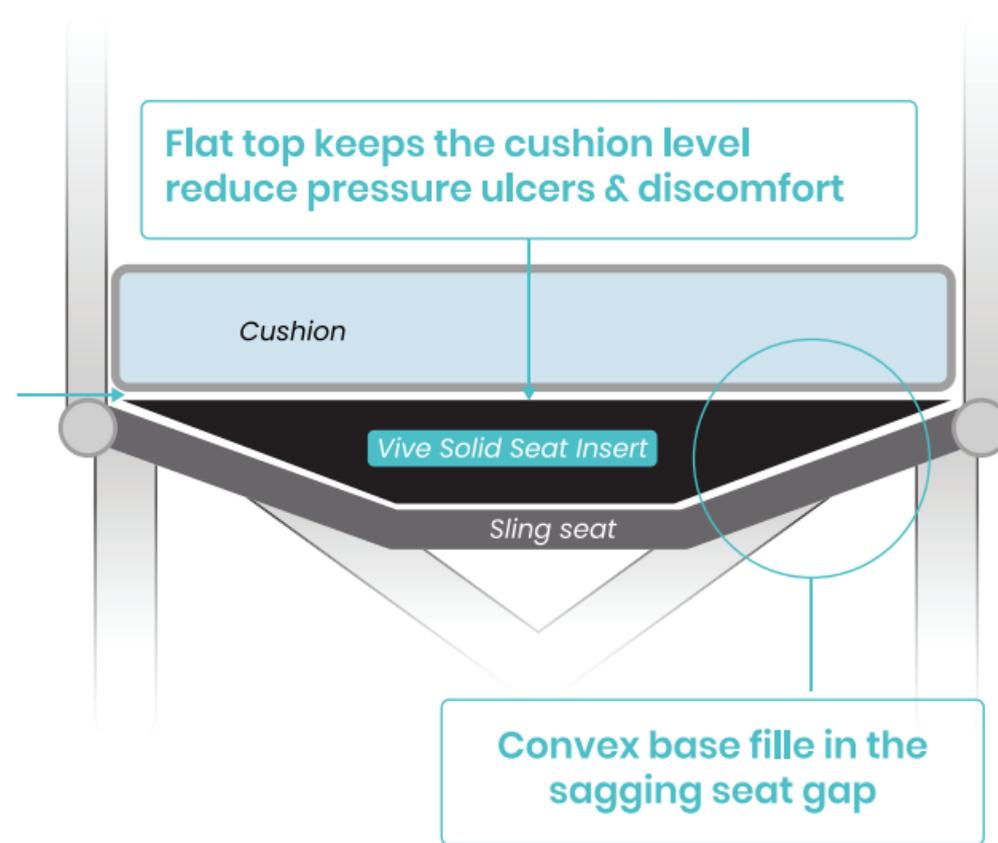

INSTALLATION

Step 1

Remove existing wheelchair cushion.

Step 2

Place seat insert with folding crease running side to side (horizontal) for best support.



Step 3

Add any cushion atop for a comfortable seating experience.



CARE

- Clean with a damp cloth or disinfectant wipe.
- Air dry completely before reapplying to wheelchair.
- Do Not Machine Wash.
- Do Not Machine Dry

WARNINGS

- Ensure the insert is properly positioned and secured before use to prevent sliding or movement.
- This product is not intended to replace prescribed pressure-relief cushions or medical devices.
- Follow all cleaning instructions to prevent damage to materials and maintain hygiene.
- Consult your healthcare provider before using this product to ensure it is appropriate for your needs.
- Store in a clean, dry place when not in use to prevent mold and mildew growth.

WARRANTY

1 year warranty against defects. Contact customer service with proof of purchase.

vive
vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com