

CATHETER BAG HOLDER

LVA3102BLK



To see all of the FAQs in one place visit vhealth.link/f50972

COMPATIBLE WITH THE FOLLOWING:

- Wheelchairs
- Walkers
- Bed Rails

INTENDED USE

The Vive Catheter Bag Holder is designed to securely conceal and carry catheter drainage bags. It ensures user privacy while allowing safe and hygienic transport of fluid bags during daily use.

APPLICATION INSTRUCTIONS

Attaching the Bag:

1. Wrap the adjustable strap over wheelchair support beam, bed rail, or walker. It may also be worn over the shoulder.



2. Secure with D-ring and Hook and Loop.

Inserting the Drainage Bag:

1. Open top of bag to insert catheter bag, then secure with hook and loop closure.



2. For added stability, use the metal hooks inside the bag to secure to top of catheter bag.



vive
vivehealth.com

QUICK START GUIDE

QUICK START GUIDE

vive

vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com

060625

3. Feed the tubing through the side zipper pipe sleeve.



4. Use the bottom zipper for discreet drainage access as needed.



CARE INSTRUCTIONS

DO'S:

- Machine wash with cold water and mild detergent.
- Lay flat to air dry.

DON'TS:

- Do not bleach.
- Do not machine dry.
- Do not iron or dry clean.

WARNINGS ⚠

- Do not overfill the drainage bag inside the holder.
- Always ensure the tubing remains unobstructed.
- Not intended to hold heavy or sharp objects.
- This product does not replace medical-grade fluid containment.

WARRANTY

The Vive Catheter Bag Holder is backed by a 1-year warranty from the date of purchase. This warranty covers defects in materials and workmanship under normal use.

If the product proves defective during the warranty period, we will, at our discretion, repair or replace the unit free of charge. Proof of purchase is required.