

FALL PROTECTION HELMET

LVA3103GRY



To see all of the FAQs in one place visit [vhealth.link/k03](https://vivehealth.link/k03)

vive
vivehealth.com

QUICK START GUIDE

INTENDED USE

This helmet is designed to reduce the risk of head injuries due to falls or sudden impact for elderly individuals, seizure-prone patients, or those with balance disorders. It is intended for indoor and supervised mobility use, not for sports or high-impact environments.

STEP-BY-STEP SETUP

1. Place helmet on head, ensuring the front edge sits above the eyebrows.
2. Secure helmet via hook and loop.



3. Wrap the chin strap under the jaw and secure the buckle.



4. Adjust the fit to ensure it is snug. The helmet should stay in place without shifting.



CARE INSTRUCTIONS

- Hand wash only using cold water and mild detergent
- Do not bleach, machine wash, tumble dry, or iron
- Lay flat to air dry in a shaded, ventilated area

vive
vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com

- Do not wring or twist the foam padding
- Store in a dry, cool location when not in use

DO's

- Use during daily routines, walking, or transfers
- Adjust chin strap for secure, comfortable fit
- Inspect regularly for wear or damage
- Consult a healthcare professional before use if needed

DON'Ts

- Do not use for biking, skateboarding, or impact sports
- Do not submerge completely in water
- Do not use if padding is compressed or damaged
- Do not leave in hot environments (e.g., parked car, near heater)

WARRANTY

This product includes a 6-month limited warranty against manufacturing defects. Warranty does not cover damage from misuse, improper care, or wear and tear from extended use. For warranty claims, please contact customer support with proof of purchase.