

BARIATRIC WALKER WITH WHEELS

MOB1133

What's Included:

- 1x Folding Walker
- 2x Adjustable Wheel Legs



To see all the FAQs,
visit vivehealth.com/7zh


vive
vivehealth.com

QUICK START GUIDE

OPERATING INSTRUCTIONS

Product is fully assembled and ready to use. (This item will require no tools to assemble.)

To begin folding the walker, simply press down on each rubber tab located on the center support bar to unlock each side. Then pull the back of each side in towards the center of the walker to fold the walker.

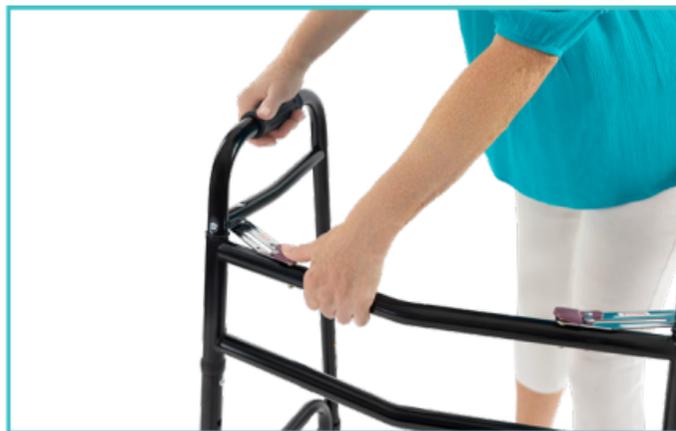

Insert the two wheels onto the front of the Walker. Note the L and R stickers to match the Left front wheel with the Left Side and Right with the Right. Insert the back two rubber tip legs on the rear side in the same fashion.



To adjust the length of the legs, press down on the rounded metal button on the leg to unlock the leg section. Then push/pull the leg to the desired length, and align the depressed button with the adjustment hole corresponding to the desired length.

vive
vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com

140725

The button will click back into place and lock the leg in the desired height. Repeat with all 4 legs and adjust to an even height that is stable and comfortable to use.



For a video tutorial on wheel and leg installation check out vhealth.link/4ku

WALKING WITH THE WALKER

- Align the middle of your foot with the back legs of the walker.
- Lift the walker up and place it a step's distance in front of you, making sure all 4 legs are stable on the ground.
- Carefully walk forward one step, and repeat.

SPECIFICATIONS

- Width - 23.5"
- Depth - 20"
- Adjustable height - 32" - 40"
- Product Weight - 13 LBS
- Max User Weight - 600 LBS

WARNING

- Do not push one side of the walker ahead and then other side without lifting the walker first.
- Use Vive accessories only. Use of other manufacturers accessories can result in improper fit and risk fall and/or injury.
- Do not use on stairs or escalators.
- For adult use only, not meant for standing or sitting on.

QUICK START GUIDE