

www.coretechortho.com

coretech[®]
by **vive health**

OWNER'S MANUAL
621 SI BELT
SKU: SUP2057BLK



To see all FAQs in one place, visit, [coretechortho.link/v4d](https://www.coretechortho.com/link/v4d)

621 SI BELT

WEARING THE 621 SI BELT

To ensure proper fit, take the patient's circumference around the pelvis, not the waist.

1. Wrap the support around your pelvis.
2. Secure the inner straps. Use the thumb loop to help pull the strap tight.
3. Use the outer string straps to tailor the fit. Set to desired compression level and secure the straps.



USING THE EXTENDER (NOT INCLUDED)

A belt extender can be purchased for individuals desiring more belt length around the pelvis.

1. Attach the extender to the hook side of the belt.



2. Wrap the belt around your pelvis and secure to a comfortable fit.

CARE

- Spot clean with a damp cloth.
- Air Dry.
- DO NOT machine wash or dry.

WARNINGS

- Consult your doctor before use of this brace.
- Stop use if pain persists or worsens.
- Assistance may be required during the application of the support.
- DO NOT over tighten the outer straps, doing so may impair blood circulation.

 **coretech**®
by **vive health**

Distributed by

**vive
health**

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com