

908 WRIST SPLINT

QUICK START GUIDE

APPLICATION INSTRUCTIONS

1. Loosen straps and slide your hand into the brace.



2. Fasten the straps from bottom to top.



coretech™


Distributed by

vive
health

8955 Fontana De|Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com
Made in China



Note: All straps can be cut to size.

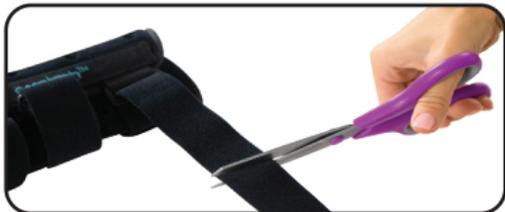

CARE INSTRUCTIONS

Remove the main splint.

Hand wash with a mild detergent in cold water. Hang dry. Do not machine wash or dry.

WARNINGS

- Consult your doctor before use.
- Stop use if pain occurs or worsens.
- Assistance may be required during the application of this support.
- Do not over tighten, doing so may lead to restricted blood circulation.
- If you experience any pain, swelling, sensation changes, or any unusual reaction while using this product please consult your doctor immediately.
- Do not machine wash or dry the brace.