

OWNER'S MANUAL  
360 WALKER BOOT SHORT  
SKU: SUP3091BLKS



SUP3091BLKS

## 360 WALKER BOOT SHORT

### PUTTING THE WALKER ON



For a video demonstration,  
check out [coretechortho.link/6ks](https://coretechortho.link/6ks)

1. Unfasten all straps and remove the soft padded liner.



2. Place your foot inside the padded liner and secure the top and bottom straps.



3. Place your foot into the boot with your heel completely set back.



4. Secure the straps from bottom to top and tighten to a comfortable fit. Do NOT overtighten.



5. If the boot is uncomfortable, loosen the straps and use the extra pads included to pad the inner liner or straps.



## SUP3091BLKS

6. Press the valve to inflate to your desired compression.



7. Press the metal button above the pump to deflate the bladder.





 **coretech**®  
by **vive health**

Distributed by

**vive  
health**

8955 Fontana Del Sol Way  
Naples, FL 34109  
1-800-487-3808  
[www.vivehealth.com](http://www.vivehealth.com)