

WHEELCHAIR BAG XL

LVA3104BLK



To see all the FAQ's,
visit [vhealth.link/103](https://vivehealth.com/103)

vive
vivehealth.com

QUICK START GUIDE

COMPATIBLE WITH THE FOLLOWING:

- Wheelchairs
- Transport Chairs
- Power Wheelchairs

INTENDED USE

The Vive Wheelchair Bag XL is designed as an accessory for mobility devices, providing extra-large, insulated storage for personal belongings, beverages, and small medical supplies while in transit.

APPLICATION INSTRUCTIONS

Attaching the bag

1. Position the bag against the back of the wheelchair and loop the top buckle straps around each push-handle.



2. Tighten the straps until the bag is secure and level.



3. Attach the strap along the inside of the wheelchair backrest for added stability.



Using the Shoulder Strap

To carry off-chair, clip the detachable shoulder strap to the D-rings on each side of the bag and adjust for comfort.



QUICK START GUIDE

vive
vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com

CARE INSTRUCTIONS

DO'S

- Machine wash on gentle cycle with cold water and mild detergent.
- Lay out flat to air dry.

DONT'S

- Do not bleach
- Do not machine dry
- Do not iron or dry clean

WARNINGS ⚠

- Ensure bag weight is evenly distributed to avoid tipping the chair.
- Do not use to transport oxygen tanks or devices that require rigid support.
- This product is not intended to prevent, cure, or treat any medical condition.

WARRANTY INFORMATION

The Vive Wheelchair Bag XL is backed by a 1-year limited warranty covering defects in materials and workmanship under normal use. If the product proves defective within the warranty period, we will, at our discretion, repair or replace it free of charge. Proof of purchase is required.

Need Help? Contact Vive health:

Phone: 1-800-487-3808

Email: service@vivehealth.com

Website: vivehealth.com