User Instructions and Warranty

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Aluminum Knockdown Shower Chair with Back

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General Warnings

To ensure the safe assembly and operation of this product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- **DO NOT** assemble and use this product without first reading and understanding this user's guide, otherwise injury or damage may occur.
- **DO NOT** use if components are missing or are damaged. Contact Medline at **1-800-MEDLINE (633-5463) | 1-800-268-2848**.
- **DO NOT** exceed weight capacity of **400 lb (181 kg)**.
- **DO NOT** make any adjustments while using the product.
- **DO NOT** stand on the product.
- **DO NOT** excessively twist or turn while seated on the chair as this may cause a loss of balance and/or damage to the product.
- **DO NOT** attempt to use the arms to support full body weight.
- **DO NOT** attempt to use the backrest to support full body weight.
- **DO NOT** use product for anything other than its intended use.
- **DO NOT** use product if wobbly, unstable, or not level.
- **DO NOT** use product on uneven surfaces.
- **DO NOT** "plop" or suddenly drop body weight onto the shower chair seat. Ease gently onto the seat when attempting to sit.
- **WARNING:** Shower chair can be slippery when wet. Use caution when attempting to sit or transfer.
- **WARNING:** Ensure weight is evenly distributed between the arm supports, if applicable, when using the shower chair.

Cautions

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:

1. Ensure chair is completely unfolded before use. Failure to do so may result in possible injury or damage.
2. All push buttons are fully protruded and are properly engaged.
3. There are no loose, broken or missing components.
4. All parts are secure.
5. Ensure the chair remains balanced.
6. Excessive twisting or turning while seated on the chair may cause a loss of balance and/or damage to the product.
7. All four legs touch the floor simultaneously at all times.

8. User evenly distributes weight between the arm supports when transferring in and out of the chair.
9. The tips are not worn, broken or missing. Replace any missing or damaged tips before use.
10. Ensure all legs are at the same height adjustment level before use.

Assembly

If exposed to cold temperatures, allow contents to come to room temperature before completing assembly. Remove contents from box, ensuring all parts listed are included:

- Backrest with 2 backrest supporting tubes (1 pc)
- Backrest connecting tubes (2 pcs)
- Seat with support tubes (1 pc)
- Leg tube with tips (4 pcs)
- Seat tube plugs (4 pcs)

Use

1. Place seat upside down on a flat surface so the four leg receptacles are facing upward.
2. Remove the black seat tube plug, then insert first leg into the leg receptacle by depressing pushbutton. When the leg is in place, the pushbutton will extend out and lock the leg in place (**Figure 1**).
3. Repeat Step 2 for the remaining three legs.
4. Insert the backrest connecting tube into backrest seat tubing by using the pushbutton (**Figure 2**).
5. Insert the backrest support tubes into the backrest connecting tube until they're fully in place.

Figure 1

Figure 2

Leg Height Adjustment

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes (**Figure 3**).
2. Ensure that the legs are locked into place by pushing down and pulling up on each leg.

Figure 3

IMPORTANT NOTE: Ensure the user's legs do not dangle while sitting on the bench as this could potentially cause a loss of circulation in the legs.

Maintenance

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at **1-800-MEDLINE (633-5463) | 1-800-268-2848**.

Cleaning

1. Clean with a non-abrasive cleaner or household cleaner with warm water.
2. Rinse with warm water and wipe dry after cleaning.
3. **DO NOT** use abrasive powders, cleaners, cloths, or scouring pads.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2025 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP.

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