GUARDIAN[®]

User Instructions and Warranty | Mode d'emploi et
garantie | Instructivo y garantía del usuario

Bedside Commode Commode de chevet

Cómod de cama

Weight Capacity
Capacité de poids de
Capacidad de peso de
350 lb (158 kg)

DO NOT install this product without first reading
and understanding this instruction manual.
FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT
IN SERIOUS INJURIES.

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General Warnings

DO NOT install this product without first reading and understanding this instruction manual.

To ensure the safe assembly and operation of this product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- **DO NOT** assemble and use this product without first reading and understanding this user's guide, otherwise injury or damage may occur.
- **DO NOT** use if components are missing or are damaged. Contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.
- **DO NOT** exceed weight capacity of:
MDS89664 - **350 lb (158 kg)**
MDS89664A - **300 lb (136 kg)**
- **DO NOT** make any adjustments while using the product.
- **DO NOT** stand on the product.
- **DO NOT** attempt to use the arms to support full body weight.

NOTE: Tips on This Product Contain Dry Natural Rubber.
FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURIES.

Installing and removing the commode seat and lid (Figure 1).

Figure 1

Caution:

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:

1. Toilet seat must be in the down position before use. Failure to do so may result in possible injury or damage.
2. All push buttons are fully protruded and are properly engaged.
3. There are no loose, broken or missing components.
4. All parts are secure.
5. When using the backrest, ensure the commode remains balanced. Excessive twisting or turning while seated on the commode may cause a loss of balance and/or damage to the product.
6. All four legs touch the floor simultaneously at all times.
7. User evenly distributes weight between the arm supports when transferring in and out of the commode.
8. Armrests are dry.
9. The tips are not worn, broken or missing. Replace any missing or damaged tips before use.
10. The wheels and wheel locks are not worn, broken or missing. Replace any missing or damaged parts before use.
11. The wheel locks are properly engaged during use.
12. Arms are unlatched and dropped to assist in transfer on or off the commode.

Assembly

If exposed to cold temperatures, allow contents to come to room temperature before completing assembly. Remove contents from box, ensuring all parts listed are included:

- Frame (1)
- Bucket w/ Lid (1)
- Splashguard (1)
- Seat and Lid (1)

Use

Leg Height Adjustment

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
2. Ensure that the legs are locked into place by pushing down and pulling up on each leg.

Sitting and Standing

1. Back up to the commode until you feel the back of your knees touching it.
2. Reach back for the armrests. Place both hands firmly on the armrests before sitting on the commode.
3. Slowly lower yourself onto the commode by bending your elbows and, if possible, your hips and knees.
4. Reverse steps 2 through 4 for getting up. Push on the armrests as much as comfortably possible. Make sure you have your balance before releasing your grasp from the commode armrests.

Over the Toilet Use

1. Lift seat of commode and remove bucket.
2. Replace with splash guard.
3. Lift lid of toilet. Position commode over toilet with splash guard in bowl and lower commode seat.

NOTE: Check to ensure that commode is level and stable before sitting.

User Instructions

Use only as instructed by your healthcare professional.

Maintenance

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.

Cleaning

1. Clean with regular household products. Use a non-abrasive cleaner with warm water. Wipe dry after cleaning.
2. **DO NOT** use abrasive powders, cleaners or scouring pads.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Medline México

01-800-831-0898 | medlinemexico.com | mexico@medline.com

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2024 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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