

vive[®]
MOBILITY



**LIGHTWEIGHT TRANSPORT
WHEELCHAIR**

**Owner's Manual
MOB1108BLU**

vivehealth.com



To see all of the FAQs in one place
visit vhealth.link/iog

FOLDING / UNFOLDING THE CHAIR



For a video demonstration, check out vhealth.link/eq

From a stable position grip front of the seat and the backrest and push them apart until the chair is fully unfolded. Check the silver lock on the back of the chair to make sure it has clicked into place.



To fold the chair back up pull up from the bottom of the silver lock. once unhinged slowing push forward on the backrest and the chair will fold upon itself. Depending on the position of the leg rest you may have to fold this out of the way first.



ATTACHING / REMOVING THE FOOT RESTS



For a video demonstration,
check out the link here: vhealth.link/f1

LOCKING THE HANDBRAKES

The parking brake is used to help prevent the Wheelchair from moving when not in use, or to prevent it from moving while sitting or standing. To engage the parking brake pull up on each of the handles below the top bar on the backrest. If you pull them up far enough they will lock. To unlock them pull up on the smaller lever attached to the larger brake handle.



MAINTENANCE

Tire Maintenance

Tires should be cleaned occasionally with a damp cloth. Replace if tires become severely worn or cracking appears. Front casters should be checked periodically for cracking and lubricated occasionally. rear wheels should be inspected at least once a month to ensure proper tightness.

General Maintenance

Wipe metal parts with a soft cloth at least once a week. All aluminum parts can be cleaned with auto wax or similar products. If exposed to moisture, dry immediately. Do not use an abrasive cleanser that will scratch the finish.

CAUTION

Please note that this chair is designed to use for transporting another individual and is not meant for self transportation. This Transport Chair is designed to safely be a stable center of gravity but it is important to try and keep a stable center of gravity of all times while performing activities that may change your center of gravity or a passengers center of gravity. Never reach out so far that is would require you to shift in the seat or bend over out of the chair.

When you have to reach for objects that require you to lean, be certain your front wheels are pointing forward as this will stabilize the chair.

Ramps

Be sure of your own capabilities, limitations and strength before attempting to push another individual up a ramp. Correct techniques on ramp negotiation should be shown to you by a qualified professional. If you must stop on an incline, avoid any and all sudden or abrupt movement when you resume movement. Because of backward angle of the chair, sudden abrupt movement could cause instability. Do not attempt to change direction when moving in incline or decline, this could result in tipping.

WARNING

- Wheelchair will support up to a 300 lb individual. Do not exceed chair weight limit.
- Do not attempt to use wheelchair without reading carefully all instructions in this manual.
- This wheelchair has not been approved as a seat that can be used within vehicles of any kind.
- Always transfer wheelchair user to a vehicle manufacturers approved seat and use safety restraints available.
- Do not operate this wheelchair on roadways, streets or any surfaces with vehicular traffic.
- Do not attempt to operate chair on inclines without proper assistance.
- Wheelchair should be on a smooth, stable and level area with the hand brakes engaged by the transporter before transferring the passenger in or out of the wheelchair.
- When transferring in or out of wheelchair, do not stand on or apply weight to foot rests.
- Do not lean over or apply weight to the top of wheelchair back. This can cause wheelchair to tip over.
- Do not stand on wheelchair.
- Abruptly changing directions while going down an incline could cause instability and/or increase the possibility of tipping over.
- It is recommended to pull a patient in reverse over any threshold rather than push them forward over the threshold.
- Always transport patients with the armrests down. Only raise the armrests when attempting to get the patient out of the chair. In doing this always keep the brakes locked.

HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at vhealth.link/iog for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



service@vivehealth.com



1-800-487-3808



vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
Made in China