

OWNER'S MANUAL
OFFLOADER ELITE

SKU: SUP3072GRY



OFFLOADER ELITE

WHAT'S INCLUDED

- OA Knee Brace
- Allen Wrench
- Philips Head Screw Driver
- Extension Hinge-stop Set (0 - 40 degrees)
- Flexion Hinge-stop Set (0 - 120 degrees)



APPLICATION



For a video demonstration,
check out coretechortho.link/cal

In order to use the brace effectively, it's important to adjust the brace to a suitable size, each strap can be adjusted to the user's leg size and comfortability level. Professional assistance may be required.

1. Sit on the edge of a chair with your knee slightly bent and foot flat on the ground.

2. Loosen and undo all straps, then place the brace on so that the hinge lines up with the outside center of your knee cap.



3. Fasten the strap labeled "1" directly under your knee cap, followed by straps labeled "2", "3", and "4". Make sure the hinge remains aligned with the knee cap while securing the straps.



4. Stand to check the tightness of the brace and walk around bending the knee to check if the brace slides or causes any discomfort. If it does, readjust the straps while seated and retest.



If the brace is still too loose and the straps are not tight enough, you can adjust the pads and straps to a smaller size.

ADJUSTING THE STRAPS



For a video demonstration,
check out coretechortho.link/9us

Each strap comes with removable Y-straps to allow for a custom fit if the straps are too long.

1. Remove the Y-straps



2. Use the folding method to test the length before cutting. Mark the location you wish to cut, fold at this point, and then reattach the Y-strap to the fold to test the adjusted length before cutting.



****If you do not wish to cut the straps, you can keep using the folding method during use****

3. Once the strap length is confirmed you can cut and reattach the Y-straps.
4. Readjust the inner foam padding as needed.



ADJUSTING THE HINGE



For a video demonstration, check out coretechortho.link/3k8

1. Use a Philips head screwdriver to loosen both screws on the outer hinge cover.
2. Place the hinge-stops provided to restrict the range of motion to your desired extension or flexion (bend). Each hinge-stop is labeled with the degrees.
3. Secure the screws back on and test.



SUP3072GRY

ADJUSTING THE UNLOADING MECHANISM



For a video demonstration, check out coretechortho.link/d05

Take the Allen wrench provided and turn the unloading dial, located above the hinge, counter-clockwise to adjust the amount of pressure unloaded.



CARE

Do NOT machine wash or dry. Spot clean with a damp cloth and immediately dry. Keep Dry.

WARNING

If you experience any pain, swelling, sensation changes, or any unusual reaction while using this product please consult your physician immediately.

coretech[®]
by **vive health**

Distributed by

**vive
health**

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com