

www.coretechortho.com

coretech[®]
by vive health

OWNER'S MANUAL
930 AFO
SKU: SUP2036



To see all FAQs in one place, visit, [coretechortho.link/r37](https://www.coretechortho.com/link/r37)

930 AFO

APPLICATION

1. Loosen the leg strap at the top, then open the foam padding.



2. Insert the bottom end of the AFO into your shoe over the existing insole.



3. Insert your foot through the top of the AFO and into your shoe. Adjust to comfort.



4. Wrap and secure the top foam pad, then loop the strap through the D-ring and secure to itself.



CARE

Spot clean the foam pad and strap. Wipe down with a damp cloth and dry immediately. Do NOT machine wash or dry.

coretech®
by **vive health**

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com