

# coretech™

## 830 SINGLE PANEL KNEE IMMOBILIZER



Owner's Manual - SUP3019



To see all the FAQs in one place  
visit [coretechortho.link/ohf](https://coretechortho.link/ohf)

## APPLICATION

**Note:** \*\*Assistance by a medical professional is required\*\*

1. Have the patient extend their leg on a flat surface, then place the support underneath their leg with the rear splints centered.



2. Wrap the immobilizer around the knee and secure using the small hook and loop tabs.



3. Place the medial and lateral stays at the midline of the leg.



4. Start with the bottom strap and move up to secure around the leg properly.



## SHAPING THE REAR SPLINT

1. Position the Immobilizer over the edge of a hard surface, such as a table, with one end flat and the opposite end hanging. The inside of the support should be facing out.
2. Use one hand to hold the support in place and with the opposite hand press down onto the end hanging from the edge to produce a bend. Adjust as needed.



## CARE

- Spot clean with a damp cloth. Hang dry.
- Do NOT machine wash or dry.

## WARNINGS

- Consult your doctor before use.
- Stop use if pain occurs or worsens.
- Do not over tighten, doing so may lead to restricted blood circulation.
- If you experience any pain, swelling, sensation changes, or any unusual reaction while using this product please consult your doctor immediately.
- Do not machine wash or dry the brace.

coretech™

Distributed by

**vive**  
**health**

8955 Fontana Del Sol Way  
Naples, FL 34109  
1-800-487-3808  
[www.vivehealth.com](http://www.vivehealth.com)  
Made in China

031122