

# PADDED U-SLING

LVA3086

**vive**  
vivehealth.com




To see all FAQs  
in one place,  
visit [vhealth.link/e5m](https://vhealth.link/e5m)

**OWNER'S MANUAL**

## PADDED U-SLING

### WHAT'S INCLUDED

- 1x Padded U-Sling
- 1x Mesh Wash Bag
- 1x Owner's Manual

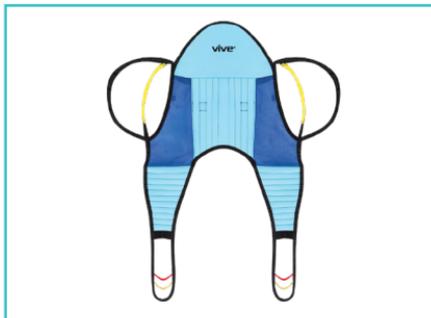

### INTENDED USE

This medical device is designed to safely transfer patients between surfaces when used with compatible patient lift systems. It provides secure support during lifting and transferring procedures while maintaining patient comfort through padded construction.

### USER PROFILE

- Healthcare professionals in clinical settings
- Trained caregivers in home care environments
- Family members who have received proper training in patient transfer techniques

## ENVIRONMENT

The medical device is intended for use in various healthcare settings. Suitable for both indoor and outdoor applications within typical residential, commercial, and occupational settings, addressing the requirements of daily living scenarios.

## SAFETY CHECKPOINTS

- Keep lift stable and in position during use.
- Ensure proper positioning of sling.
- Verify all straps are secure before and during lift.
- Maintain clear communication with patient throughout process.
- Monitor patient comfort throughout use.

## INSTRUCTIONS

1. Position the sling behind the patient while they are seated.
2. Center the U-shaped base at the patient's tailbone.



3. Guide the leg supports under each thigh.
4. Cross the leg straps for additional security if required.



5. Attach the color-coded straps to the lift in matching pairs.



6. Double-check all connections before beginning the transfer.



## SPECIAL STORAGE & HANDLING CONDITIONS

- Store in a cool, dry place when not in use.
- Keep away from direct sunlight.
- Avoid exposure to extreme temperatures.
- Inspect the sling and all components before each use.

## CARE INSTRUCTIONS

Remove plastic back panel before washing. Place in the included mesh bag, machine wash cold. Hang dry. Do not machine dry. Spot clean with a damp cloth and mild detergent. Air dry.

## WARNINGS ⚠

- Consult a doctor before use.
- Inspect patient slings prior to each use.
- Do not exceed the maximum weight capacity of the sling or patient lift.
- If you experience any swelling, discomfort, or pain, stop use immediately.
- Bleached, torn, cut, frayed, or broken slings are unsafe and could result in injury. Discard immediately if damaged.
- Do not alter the sling.

## MANUFACTURE AND PRODUCT INFORMATION

### Manufacturer



#### **Vive Health**

8955 Fontana Del Sol Way  
Naples FL 34109

service@vivehealth.com  
1-800-487-3808  
vivehealth.com



#### **Compliance Management Solutions Europe**

2 Bulgaria Str, 2850 Petrich, Bulgaria.

## SIGNS AND SYMBOLS

|   |  |
|---|--|
|    | <p>Medical device indication ISO/DIS 15223- 1:2020<br/>Reference no. 5.7.7 Indicates the item is a Medical Device.</p>   |
|    | <p>CE marking Communauté Européenne<br/>Product traded on the extended Single Market in the European Economic Area (EEA); has been assessed to meet high safety, health, and environmental protection requirements.</p>  |
|    | <p>Consult instructions for use ISO 7000-1641<br/>Indicates the need for the user to consult the Instructions for Use.</p>   |
|    | <p>Manufacturer ISO 7000-3082<br/>To identify the manufacturer of the Medical Device.</p>  |
|    | <p>Date of manufacture ISO 7000-2497<br/>To indicate the date on which the Medical Device was manufactured.</p>  |
|    | <p>Importer<br/>To identify the importer of the Medical Device.</p>  |
|    | <p>Caution ISO 15223-1, Clause 5.4.4<br/>To indicate the need for the user to consult the Instructions for Use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the Medical Device itself.</p> |
|    | <p>LVA3086</p>   |
|    | <p>Indicates a medical device that has not been subjected to a sterilization process.</p>  |
|   | <p>Latex Free</p>  |
|  | <br><small>0150081017888185732411LVA3086BLU</small>   |

## IMPORTANT SAFETY NOTICE!!

If you experience any serious incident related to the use of this device, immediately discontinue use and report it to:

1. The manufacturer: Vive Health (8955 Fontana Del Sol Way, Naples FL 34109)
2. Contact your national health authority or medical device regulator.
3. Your healthcare provider.

A serious incident is any incident that directly or indirectly led, might have led, or might lead to:

- Death of a patient, user, or other person.
- Temporary or permanent serious deterioration of a patient's, user's, or other person's state of health.
- A serious public health threat .

For any questions or concerns that are not serious incidents, please contact Vive Health customer service at:

service@vivehealth.com  
1-800-487-3808  
vivehealth.com

## WARRANTY

6 month warranty from date of purchase, excluding damage from improper use, cleaning, exceeding weight capacity, alterations, or normal wear and tear.





## HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vhealth.link/e5m](https://vivehealth.com/faq) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website

[service@vivehealth.com](mailto:service@vivehealth.com)  
1-800-487-3808  
[vivehealth.com](https://vivehealth.com)

Distributed by

The logo for Vive Health, with "vive" in a bold, teal, lowercase sans-serif font and "health" in a teal, lowercase sans-serif font below it.

**vive**  
health

8955 Fontana Del Sol Way  
Naples, FL 34109  
Made in China