

WIRELESS ALARMS

LVA2063

vive
vivehealth.com



Check out this
how-to video at
vhealth.link/8v7

OWNER'S MANUAL

WIRELESS ALARMS

INTENDED USE

Vive Wireless Alarms alert caregivers when someone gets up from bed, out of bed, or out of the chair. It helps keep patients safe.

WHAT'S INCLUDED



- Vive Wireless Sensor Pad
- Wireless Transmitter
- Wireless Pager
- 2x AA Batteries (for pager)
- Quickstart Guide

IMPORTANT SETUP ORDER

WARNING: You must connect the pager to the transmitter **FIRST**, before connecting the transmitter to the pad.




STEP 1: SET UP THE PAGER

Install Batteries

1. Open the battery door on the pager
2. Insert 2 AA batteries
3. Close the battery door



Turn On Pager

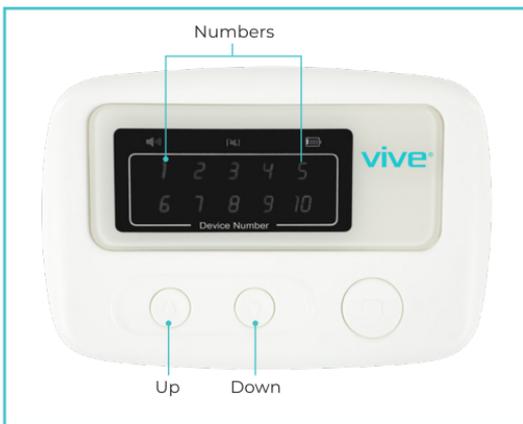

1. Press and hold BOTH the UP and DOWN buttons together.
2. Hold for 5 seconds.
3. The number "1" will start blinking in red.
4. The pager is now ready to pair.

Choose a Channel (Optional)

- The pager has 10 channels (1-10)
- Press UP button to change channels
- Each channel can pair with multiple devices
- We suggest using channel 1

STEP 2: PAIR THE TRANSMITTER

WARNING: The transmitter must NOT be connected to the pad yet!

1. While the pager shows a blinking number (in pairing mode)
2. Press and hold the transmitter button for 5 seconds.
3. The alarm light on transmitter will flash orange.
4. Release the button.



5. The pager will beep once - pairing is complete.
6. All lights on transmitter will turn green briefly.

****Connect another device or continue to finish your setup****

EXIT PAIRING MODE

Press and hold BOTH UP and DOWN buttons on pager for 3 seconds. The blinking number will stop.

STEP 3: CONNECT AND TEST

Connect Transmitter to Pad

Plug the transmitter cable into the sensor pad.
The connection will click into place.



Test the Alarm

1. Press down on the pad with your hand for 3 seconds.
2. Release your hand.
3. The pager should alarm.
4. Press STOP button on pager to silence.



DAILY USE

How It Works

- Person sits/lays on pad = No alarm
- Person gets up from pad = Alarm sounds
- Press STOP on pager to silence alarm

Multiple Alarm Tip: If you connect multiple alarms to your pager, write down which number goes with each room or patient. When the pager alarms, you'll see which number is alerting. This tells you which room needs attention.

Positioning

- **Floor Alarms**
 - Place on the side patient exits from
 - On a flat and stable surface
 - Do not cover with rugs



- **Bed Alarms**

- Place under sheet where patient lies
- Center pad under torso area for best detection



- **Chair Alarms**

- Position on chair seat where patient sits most often
- Make sure pad lies flat with no wrinkles or folds




Alarm Range

- Works up to 150 feet in open space
- Range is shorter through walls
- Keep pager within range

NOTIFICATION SETTINGS

The pager has 3 alert options:

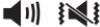Mode	Alert Type	Icon
Sound only	Beeps	
Vibrate only	Vibration	
Sound + Vibrate	Both	

To Change Alert Mode:

1. Press and hold STOP button and DOWN arrow for 5 seconds.
2. Press UP or DOWN to select mode.
3. Press STOP to save.



CARE INSTRUCTIONS

Sensor Pad

- Wipe with damp cloth
- Do not soak or submerge
- Let dry before use
- Replace after 1 year of use

Pager & Transmitter

- Wipe with dry cloth
- Do not get wet
- Check batteries monthly

TROUBLESHOOTING

Problem	Solution
Won't pair	Make sure transmitter is NOT connected to pad during pairing Check batteries in pager
No alarm when getting up	1. Check pad is plugged into transmitter 2. Test pager is in range 3. Re-pair devices
Battery light is on	Replace pager batteries
"Change Alarm" light is on	Transmitter needs replacing (after 2 years)
Alarm won't stop	1. Press STOP button firmly 2. Remove batteries and retry setting the alarm off to check if the sounds issue is still happening

WARNINGS

IMPORTANT SAFETY INFORMATION

- This device will not prevent falls.
- Not a substitute for proper supervision.
- Check the person often.

BATTERY WARNINGS

- Use only AA batteries
- Remove batteries if storing device
- Keep batteries away from children

WARRANTY

Your Vive Wireless Alarm has a 1-year warranty from purchase date. Keep your receipt. Warranty covers defects in materials and workmanship. Normal wear is not covered.



HAVE MORE QUESTIONS?

Check out this how-to video at vhealth.link/8v7



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by

The logo for Vive Health, featuring the word "vive" in a bold, teal, lowercase sans-serif font, with "health" in a smaller, teal, lowercase sans-serif font directly below it.

8955 Fontana Del Sol Way
Naples, FL 34109
Made in China