

# WIRELESS ALARM

LVA2063



Check out this how-to video at [vhealth.link/5ac6f](https://vhealth.link/5ac6f)

**vive**  
vivehealth.com

## QUICK START GUIDE


### IMPORTANT: SETUP ORDER

The transmitter **MUST** be disconnected from the pad during pairing. Only connect after pairing is complete.

### HOW TO PAIR YOUR DEVICES

#### Step 1: Start Pager Programming Mode

1. Press and hold BOTH the UP and DOWN buttons on the pager for 5 seconds. The number "1" will blink in red - the pager is ready.

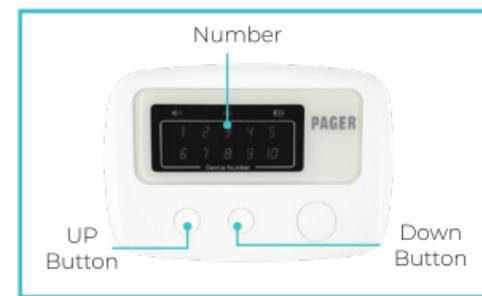

#### Step 2: Pair the Transmitter



With transmitter **NOT** connected to pad:

- Press and hold the transmitter button for 5 seconds
- Orange light will flash on transmitter
- Release the button
- Pager will beep once = successful pairing

#### Step 3: Exit Programming Mode

Press and hold UP and DOWN buttons together for 5 seconds to exit.

**vive**  
vivehealth.com

Distributed by

**vive**  
health  
8955 Fontana Del Sol Way  
Naples, FL 34109  
1-800-487-3808  
[www.vivehealth.com](http://www.vivehealth.com)

## Step 4: NOW Connect and Test

- Plug transmitter cable firmly into sensor pad until it clicks.
- Press down on pad for 3 seconds, then release.
- Pager should alarm with channel number flashing.
- Press STOP button to silence.



## PLACEMENT BEST PRACTICES

### Pad Positioning

- **Floor Alarms**
  - On the side patient exits from
  - On a flat and stable surface
  - Do not cover with rugs



- **Bed Alarms**
  - Place under sheet where patient lies
  - Center pad under torso area for best detection



- **Chair Alarms**
  - Position on chair seat where patient sits most often
  - Make sure pad lies flat with no wrinkles or folds



- **Transmitter Location**
  - Mount within easy reach of caregiver
  - Keep cable away from patient's movement area
  - Secure cable to prevent tripping
- **Pager Placement**
  - Keep within 150 feet of transmitter
  - Clip to belt or place in pocket
  - Test range in your specific facility

## TO SILENCE ALARM

When triggered, the channel number flashes on pager. Press STOP button firmly to silence.