

UNDER SEAT WHEELCHAIR HARNESS

LVA3114BLK



To see all the FAQs,
visit [vhealth.link/i9c](https://vivehealth.com/faq)

vive
vivehealth.com

QUICK START GUIDE

APPLICATION

Step 1

Lock the wheelchair brakes and ensure the patient is seated upright.

Step 2

Place the harness over the patient's head, allowing the top strap to rest along the back of the wheelchair.



Step 3

Wrap the side straps around the backrest of the wheelchair, making sure they are positioned evenly on both sides. Adjust as needed.



Step 4

Connect the bottom strap from beneath the seat to the top strap at the back of the wheelchair. Tighten or loosen to achieve a secure fit.



vive
vivehealth.com

Distributed by

vive
health
8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com

Step 5

Confirm the patient is comfortably supported and stable. Make final adjustments to ensure proper positioning.



CARE INSTRUCTIONS

- Machine wash cold with mild detergent.
- Air dry only. Do not bleach, iron, or tumble dry.
- For spot cleaning, wipe with damp cloth.
- Inspect straps and stitching regularly. Replace if worn or damaged.

MATERIAL

- 100% Polyester (Straps + Body)

WARNINGS

- **Not a Restraint Device** – Intended only for positioning and support. Do not use as a restraint or for transport.
- **Supervised Use Only** – Must be used under the supervision of a caregiver or medical professional.
- **Not for Vehicle Use** – Not designed for use in automobiles, buses, or any moving vehicle.
- **Check Fit Regularly** – Ensure straps are properly adjusted and not too tight to restrict breathing or circulation.
- **Inspect Before Each Use** – Do not use if frayed, torn, or missing stitching.
- **Do Not Overtighten** – May cause restricted breathing or discomfort.
- **Do Not Leave Patient Unattended** while using the harness.

WARRANTY INFORMATION

Vive Health Limited Warranty – 90 Days

This warranty covers defects in materials and workmanship under normal use and service.

What is Covered:

- Manufacturing defects and workmanship issues.
- Replacement or repair of harness at no additional cost if deemed defective.

What is Not Covered:

- Normal wear and tear, including fabric fraying, fading, or stitching wear.
- Accidental damage caused by misuse, improper care, or modification.
- Use outside intended purpose (positioning support for wheelchair users).

LIABILITY DISCLAIMER

Vive Health is not responsible for injury or damage resulting from misuse, failure to inspect before use, or using the harness without proper training or supervision. Always use under the guidance of a qualified caregiver or medical professional.