

# 807 THUMB SPLINT

## QUICK START GUIDE

### APPLICATION INSTRUCTIONS

1. Loosen all straps and carefully insert your hand so that the thumb is positioned into the splint.



2. Tighten the straps from bottom to top.



Note: All straps can be cut to size.



### CARE

Remove all splints, hand wash with mild detergent. Air Dry

### WARNINGS

- Consult your doctor before use.
- Stop if pain worsens.
- If you experience any pain, swelling, sensation changes, or any unusual reaction while using this product please consult your doctor immediately.
- Do not over tighten, doing so may lead restricted blood circulation.