

WALKER SKI GLIDES HEAVY DUTY

MOB1118



To see all the FAQ's,
visit [vhealth.link/ef3](https://vivehealth.com/ef3)

vive
vivehealth.com

QUICK START GUIDE

WHERE TO INSTALL SKI GLIDE

- Can be installed on front OR back legs (Typically installed on the back legs)
- For walkers with front wheels: Install on back legs
- For walkers without wheels: Can install on all 4 legs
- **IMPORTANT:** Orient curved part in walking direction



INSTALLATION GUIDE (FOR CAREGIVERS)

No tools Required!

1. Turn walker upside down with legs pointing up



2. Gently twist and remove existing rubber feet

3. Install ski glides on chosen legs
- Make sure curved part points forward



4. (Optional) Add fabric covers for wood floors
5. Flip walker right-side up

6. Press down gently to ensure:
- All legs are level
 - Ski glides are fully attached



SAFETY TIPS

- Ensure ski glides are securely attached and do not wobble before each use.
- Do not use on stairs
- Be careful on wet surfaces
- Replace if damaged or worn

WARRANTY

Covered by 1-year warranty against defects.
Contact customer service at 1-800-487-3808
or service@vivehealth.com for support.

vive
vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com