

Fitting & Care Instruction

Model: DCT-MTBF

Instructions on putting on brace:



Step 1: Open the back flaps on the back of the brace and open the alligator Velcro clamps to release the side wings on both sides.

Step 2: Remove the left and right wings.

Step 3: Find your desired size and fold to the needed length. Make sure to do both the left and right side the same.



Step 4: If there is too much extra fabric cut after one of the Velcro strips. Note: make sure to leave length in case you need to adjust the sizing

Step 5: Place both wings back in to the alligator clamps and close the back flaps. Center the back of the brace at the center of your back. Note: The posterior panel can be removed if needed.

Step 6: Wrap the left wing around your waist. **Make sure to wrap under the belly not across the belly.**



Step 7: Wrap right wing on top of the left wing and secure to the front underneath the belly.

Step 8: Locate the rubber pull tabs and tighten to desired support angled underneath the belly.

Step 9: Attached the rubber tabs to the front of the brace. The front closure should sit comfortably under the belly

Care Instructions: Hand wash with a light detergent.

Warranty: 6 month warranty from any defects.