

TOILET SAFETY RAIL

LVA1010

vive
vivehealth.com



To see all FAQs in
one place,
visit vhealth.link/eef06

OWNER'S MANUAL

TOILET SAFETY RAIL

WHAT'S INCLUDED



- 1x Rear Bracket (A)
- 2x Armrests (B)
- 2x Slide Assist Rails (C)
- 2x Backrest Screw Knobs (D)
- 2x Screw-on Suction Leg Tips (E)
- 2x Armrest Knobs (F)
- 2x Armrest Screws (G)
- 2x Armrest Washers (H)



ASSEMBLY INSTRUCTIONS



for an easy instructional video
Check out vhealth.link/wsv

1. Detach the toilet seat lid from the toilet where you will be using this product.
2. Locate the rear white bracket (A) with the two screw holes on the toilet safety rail.
3. Position the screw holes on the bracket (A) to the screw holes on the toilet, making sure the bracket is flat against the toilet.
4. Add the toilet seat lid on top of the rear bracket (A), align all the holes and fasten with the same screws and bolts.
5. Screw suction leg tips (E) onto the ends of armrests (B).



6. Attach Armrest (B) to Side Assist Rail (C) by first placing Screw (G) into the hole to connect the two pieces (Make sure the hex head of the screw fits flush into the hex hole).



Then on the opposite end of the screw place the washer (H) and the knob (F) and tighten. After Completing Step 7. Repeat steps 6 and 7 for the opposite side of the device.



- Slide rear assist rails (C) into rear bracket (A). Set arm rests to desired width. Use Backrest Screws Knobs (D) to secure in place.



- Adjust assist rail leg extensions according to the height of the toilet.



WARNINGS ⚠

- Check periodically to make sure all fixings are tight and secure.
- **Max User Weight: 300 LBS**

HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at
vhealth.link/eef06 for helpful answers.



And if that doesn't answer your question, our
customer service team would love to help!
Feel free to connect with them by phone,
e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com