**Bariatric Rollator, Heavy Duty
Rolling Walker with Wheels, Steel**
Déambulateur, roulant très résistant, acier
**Andador de paseo de servicio pesado con
ruedas, acero**

Use Instruction and Warranty Guide

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Safety Instructions

General Warnings and Cautions

- **DO NOT** use this product without first reading and understanding the instructions contained in this manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional or dealer before use. Failure to do so may result in bodily injury or damage to the device.
- **DO NOT** self-propel the rollator while seated. Rollators are intended for individual use only and are **NOT TO BE USED AS A WHEELCHAIR**. Serious injury to the user and/or damage to the rollator may result from improper use.
- **DO NOT** use the rollator to walk backwards, down gradients or to climb stairs. Avoid curbs and other obstacles as serious risk of fall or injury may occur.
- All wheels must be in contact with the floor and/or ground at **ALL TIMES**.
- **DO NOT** exceed the maximum weight capacity (including bag contents) or serious injury could result.
Maximum weight capacity: **500 lb (226 kg)**
Place items completely in the bag/basket so they do not protrude. Excess weight may cause the rollator to tip, resulting in bodily injury or damage to the rollator. The bag/basket has a weight limitation of **5 lb (2.2 kg)**.
- **DO NOT** perform any adjustments to the rollator while it is in use.
Before each use make sure the rollator frame is fully and securely open.
Observe the minimum insertion depth of the push handles.
Only use accessories and replacement parts authorized by Medline Industries, LP.

Seat Warnings

- The Brakes **MUST** be in the locked position **BEFORE** sitting on the seat.
- **DO NOT** attempt to move the rollator while you or anyone is sitting in the seat.
- **DO NOT** sit on the rollator while it is being loaded into a vehicle or while it is being transported in a vehicle .
- **DO NOT** use the seat to transport people or objects.
- **DO NOT** sit on the seat when the rollator is on an incline.
- The rollator was designed to be used as a walking aid. **DO NOT** use the rollator in a manner that could cause personal injury; such as a scooter, a knee walker or a jumping/hopping aid.
- **DO NOT** stick fingers or hands near the seat slide, folding mechanism or any moving parts. Use caution when folding and unfolding the rollator. Make sure that clothing and body parts do not get pinched between moving parts.

- **CAUTION:** The backrest is **NOT** intended to support full body weight. Avoid leaning while sitting as this may result in a fall.
- Use **EXTREME** caution when shifting weight while seated, as this may cause the rollator to tip.
- Check that the rollator is fully intact each time before using. Check that the brakes are in good working order prior to sitting. The rear wheels should not rotate freely if the brakes are engaged.
- **DO NOT** “plop” or suddenly drop body weight onto the rollator seat. Ease gently onto the seat when attempting to sit.

Brake Warnings

- Check the brakes for proper operation prior to each use. Make sure that all parts are secure and that the moving parts are in good working order. The rear wheels should not rotate freely if the brakes are engaged.
- **DO NOT** use the rollator to provide assistance to the user when rising from a lower position. The rollator may move even if the brakes are in the locked position and result in a fall.
- **DO NOT** push the rollator while the brakes are locked. Even if the brakes are locked it may not prevent the rollator from moving if pushed on certain surfaces.

Environmental Warnings

- Contact with water or excess moisture may cause parts to rust or corrode.
- **DO NOT** use the rollator in a shower, pool or other body of water. Avoid excess moisture (do not leave the rollator in the bathroom while taking a shower).
- If the rollator is exposed to extreme temperatures, high humidity and/or becomes wet, check parts are in good working order and that the handgrips are not loose and cannot be twisted.
- **DO NOT** push the rollator while the brakes are locked. Even if the brakes are locked it may not prevent the rollator from moving if pushed on certain surfaces.
- **DO NOT** use or store the rollator near open flames or combustible materials.
- The surfaces of the rollator can heat up when exposed to direct sunlight or used on hot surfaces that could result in product damage and/or personal injury.

What's Included:

I

Orientation of Rollator

Front

- A** 6 Triangular Knobs
- B** Bag
- C** Rollator Frame
- D** 2 Handle Bars with Brake Assembly

- E** 2 Rear Wheels
- F** 2 Front Wheel Assemblies
- G** Folding Strap
- H** Backrest

- I** Use Instruction and Warranty Guide

Examine all parts for shipping damage. **DO NOT** use the rollator if there is apparent damage or if any parts (other than the bag) are missing. If there are missing or damaged parts, please call Medline Customer Care at 1-800-MEDLINE, Medline Canada 1-800-268-2848.

This guide applies to the following rollator:

Item Number	Description	Frame Color	Weight Capacity
MDS86800XWS	Bariatric Rollator, Heavy Duty Rolling Walker with Wheels	Burgundy	500 lb (226 kg)
MDS86800XWBS	Bariatric Rollator, Heavy Duty Rolling Walker with Wheels	Blue	500 lb (226 kg)

First: Open the Rollator

Remove the rollator components from shipping carton. Then, in an upright position, unfold the rollator frame (Part C) by pushing down on the folding bracket until the frame is in full extension.

IMPORTANT NOTE:

Proceed with caution when opening the rollator as there are areas around the moving parts that could potentially pinch fingers.

Place the rollator on its side.

Install the Wheels

Front Wheels

Prior to assembly, remove four black shipping plugs from tubing frame and discard.


Attach the front wheel assemblies (part F), which do not have brakes, to the front of the rollator. Ensure the support bar on the wheel aligns with the slot on the frame.


Use the triangular knob (part A) to screw the front wheel assembly in place. **Hand tighten ONLY!** Repeat steps 1-3 for the other front wheel assembly.

Rear Wheels & Brakes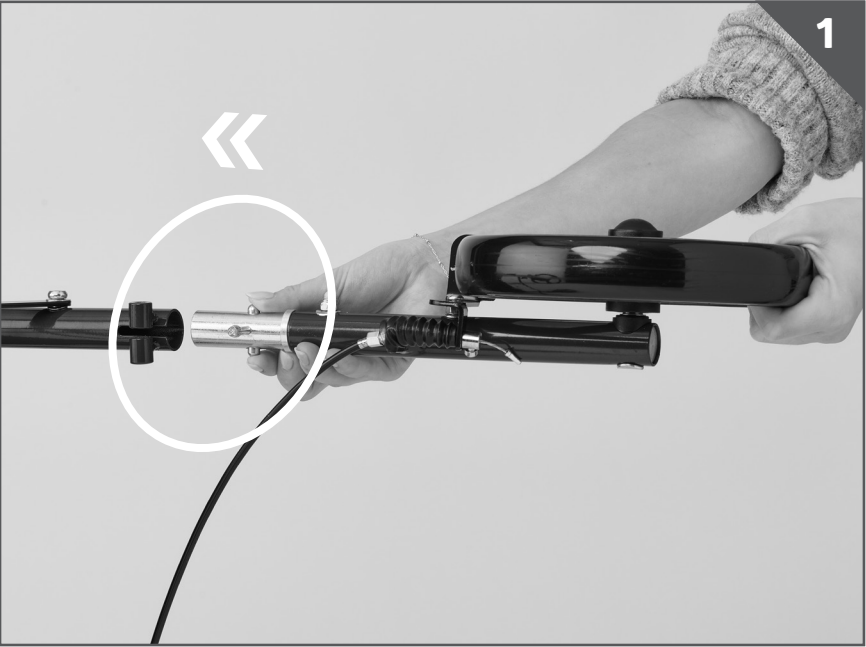

Locate the handle bar with brake assembly (part D). With the rollator on it's side, attach rear wheels with brakes. Ensure the support bar on the wheel aligns with the slot on the frame, and that the rear wheels and brake shoe are on the outside of the rollator.

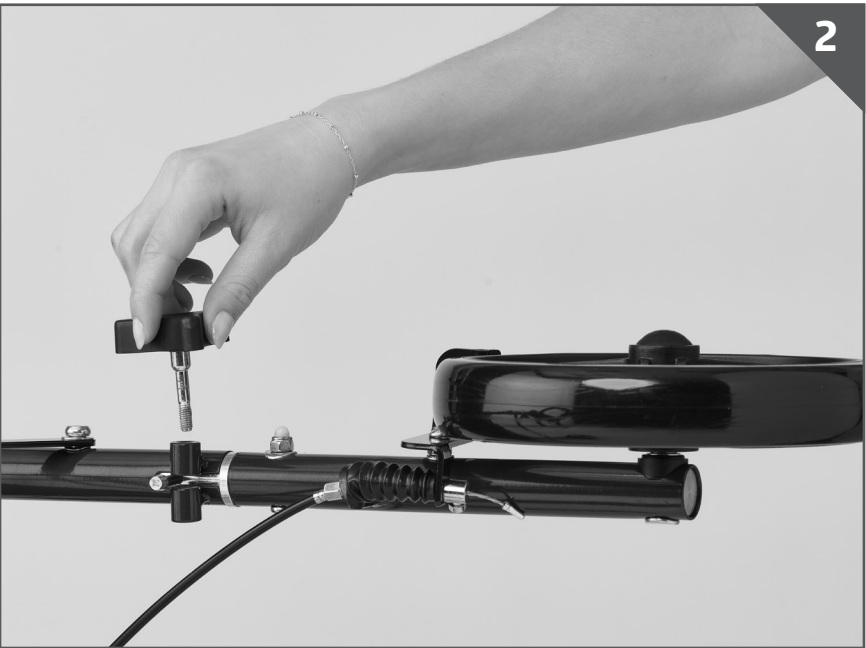
Use the triangular knobs to thread the rear legs in place.
Hand tighten ONLY. Ensure parts are secure prior to each use.
Repeat on other side.

Handles


Insert the handle assembly into the frame and set to desired height by lining up the holes. Brake cables should be on the outside of the handle bars and frame.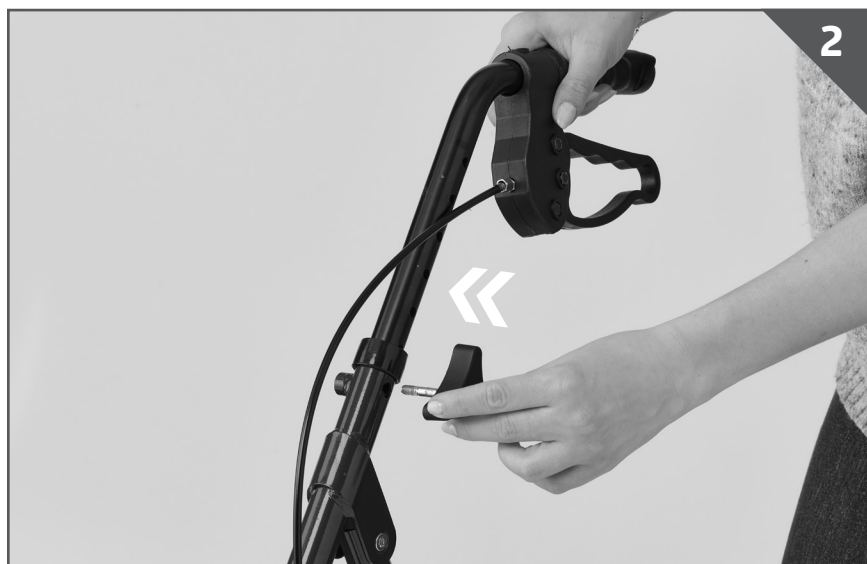


Use triangular knob to secure. **Hand tighten ONLY!** Ensure parts are secure prior to each use. Repeat on other side.

- 3 CAUTION:** Ensure both push handle assemblies are at the same height prior to use. **DO NOT** use the rollator if the handles are set to different heights.

Finally Install Backrest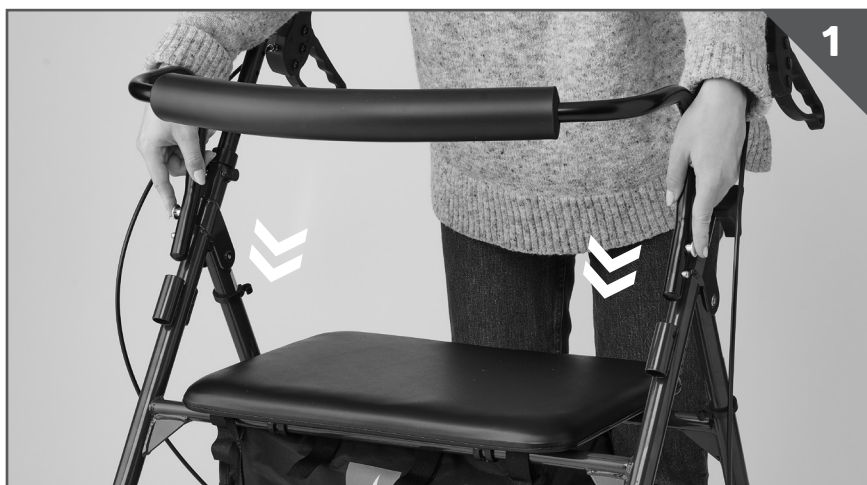

Stand the rollator up so all 4 wheels are on the floor. Insert the backrest (part H) so that the curved portion faces the front of the unit. Make sure that the button snaps firmly into the hole so the back is secure.

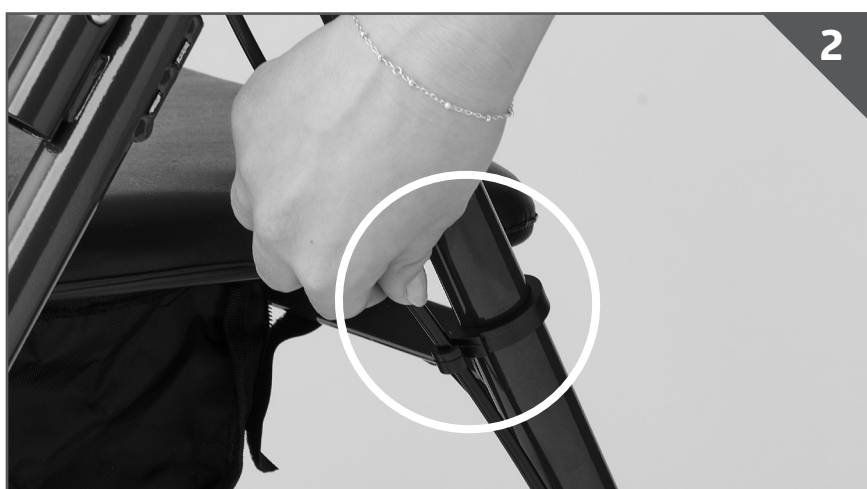Attach a brake clip to the rear frame tube. Attach the brake cable into the brake clip. If brakes are too tight during assembly, fully extend brake cable to release tension and re-attach to brake clip. Then, repeat this step on the other side of the rollator.

Attaching the Bag

The bag is attached to the rollator prior to shipping. If the bag has come unattached in shipping, loop the cloth hooks onto the seat frame and make sure the bag is centered.

Operating Instructions

Activating the Brakes

Running Brake

To activate the running brake, squeeze both handbrake levers upward on the push handles. Running brakes act on the back fixed wheels.

Parking Brake

- To lock the parking brakes, press down on both brake levers until an audible click is heard.
- To release the parking brake, pull both brake levers back into the middle position.

How to Fold Rollator

To fold, pull up on the folding strap (part G).

How to Unfold the Rollator

With the rollator in an upright position, unfold the rollator by pushing down on the folding bracket until the frame is in full extension.

IMPORTANT NOTE: Proceed with caution when unfolding the rollator as there are areas around the moving parts that could potentially pinch fingers.

Walking

- Use on a level, firm surface.
- Familiarize yourself with the function of the brakes.
- Place yourself in the rollator so your ankles are at the height of the rear wheels.
- Stand with good posture.
- **DO NOT** slouch or lean onto the push handles.
- Grasp the push handles with both hands and your arms bent at a slight angle.
- When walking, try to stay as close as possible to the push handles. Try to walk in the rollator and not behind it. Take small steps and **DO NOT** push the rollator too far ahead.
- Avoid having to lift the rollator over obstacles. Use ramps and go around obstacles whenever possible. Use assistance, if avoiding obstacles is not possible.

Using the Seat:

- Lock Parking brakes **BEFORE** sitting on the seat.
- Use extreme caution when maneuvering to sit or to stand.
- Turn your body so that push handles are at your sides and you are facing away from the backrest.
- Carefully lower onto the seat as the rollator may move even if the brakes are in the locked position.
- When rising to a standing position, use extreme caution to not push the rollator away.
- Brakes **MUST** remain locked while seated.

Technical Data

Type	Bariatric Rollator, Heavy Duty Rolling Walker with Wheels
Height Range of Push Handles	31.7" – 37.8"
Seat Dimensions	13" x 17"
Seat Height	22" from ground to seat top
Rollator Weight	24 lb (10.9 kg)
Weight Capacity of Rollator	500 lb (226 kg)
Weight Capacity of Bag/Basket	5 lb (2.2 kg)

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE (633-5463) | Medline Canada 1-800-268-2848.

Cleaning Instructions:

If dust or dirt accumulates on the rollator, follow the cleaning instructions below.

- The frame of the rollator can be cleaned with a moist cloth with mild detergent.
- The wheels can be cleaned with a wet synthetic fiber brush (**DO NOT** use wire brush).
- Push handles, seat and backrests can be wiped down using mild detergent.
- **DO NOT** use aggressive or caustic detergents to clean the rollator.
- Periodically wipe down the rollator with a soft cloth and non-abrasive cleaner.
- Dry completely before use.

Brake Inspection and Adjustment:

- **DO NOT** use the rollator if the brakes are not functioning properly.
- Make sure your brakes are operating properly before **EACH** use.
- If the brakes are not functioning properly, discontinue use immediately and **DO NOT** use the rollator.
- Contact Medline customer care at 1-800-MEDLINE, if assistance is required.

Adjusting the Brakes

Scan the QR code below for video instructions

Contact Medline Industries, LP, if assistance is required to adjust the brakes.

- To adjust the brakes, stand behind the rollator, press the brake shoe onto the wheel and pull the cable upwards.
- While still pressing the brake shoe to the wheel, turn the silver nut to clockwise to decrease distance between the wheel and brake shoe. Turn the silver nut counter clockwise to increase the distance.
- Release the brake shoe and test the brake makes contact with the wheel and does not allow for the wheel to spin. Make sure the brake shoe **DOES NOT** contact the wheel, unless the brakes are activated.
- Repeat on the other brake, if necessary.

CAUTION: Prior to each use, check that the brakes are functioning properly, that all parts are secure, and that all moving parts are in good working order. The rear wheels should not rotate freely if the brakes are engaged.

Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This limited warranty is an expression of our confidence in the materials and workmanship of our products and offers convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as replaceable components, accessories, casters and grips, which are subject to normal wear and replacement.

Limited Lifetime Warranty on the Frame, 6 months on:

- Brakes
- Brake components (Hand Brake, Brake Cable)
- Adjustment Knobs
- Push-Buttons
- Backrest
- Upholstery
- Wheels (Front and Rear)
- Bag/Basket
- Other Accessories

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©2025 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP.
©2025 Medline Industries, LP. All rights reserved/tous droits réservés/todos los derechos reservados. Medline is a registered trademark of/est une marque déposée de/es una marca comercial registrada de Medline Industries, LP, Three Lakes Drive, Northfield, IL 60093 USA.