

ELASTIC POSTURE CORRECTOR

SUP3006

vive
vivehealth.com



To see all FAQs
in one place,
visit [vhealth.link/lfm](https://vivehealth.link/lfm)

OWNER'S MANUAL

ELASTIC POSTURE CORRECTOR

APPLICATION

1. Adjust the length of the straps to suit your size and preferences.



2. Slide your arms into the loops of the corrector and pull over your shoulders like a backpack. Align the backside with your spine.



3. Secure the main straps at the front under the chest.



4. Secure the small fastening straps on either side of your body for extra support and compression.



CARE INSTRUCTIONS

- Hand wash cold with mild detergent. Air Dry.

WARNINGS

- Consult your doctor before use.
- Stop use if pain worsens.
- Assistance may be required during the application of this support.
- Do not overtighten. Doing so may lead to restricted blood circulation.
- If you experience any pain, swelling, sensation changes, or any unusual reaction while using this product please stop use and consult your doctor immediately.

HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at
[vhealth.link/lfm](https://vivehealth.com/faq) for helpful answers.



And if that doesn't answer your question, our
customer service team would love to help!
Feel free to connect with them by phone,
e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com