

MATERNITY BELT

SUP3085



vive
vivehealth.com



To see all FAQs
in one place,
visit vhealth.link/rcm

OWNER'S MANUAL

MATERNITY BELT

WHAT'S INCLUDED

- 1x Maternity Belt
 - Abdominal Support Pad
 - Back Support Brace
 - Elastic Tummy Strap
- Owner's Manual



INTENDED USE

The medical device is designed to provide support, compression, and/or immobilization for musculoskeletal injuries, post-operative recovery, and chronic conditions. It is intended to aid in the relief of pain, reduce swelling, and promote healing of specific body areas. The device may be used under the guidance of a healthcare professional for rehabilitation, pain management, or daily support.

USER PROFILE

Primary Users: Pregnant individuals seeking support for their lower back, abdomen, and pelvis during pregnancy.

Age Range: Typically between 20–40 years, though it can vary based on individual circumstances.

ENVIRONMENT

The maternity belt is designed for use in everyday environments, including at home, work, or during light physical activities. It is suitable for indoor and outdoor use, provided the user avoids strenuous activities while wearing it. Ensure the environment is clean and dry when putting on the belt to maintain hygiene and functionality.

PERFORMANCE CHARACTERISTICS

- **Support Level:** Provides medium to firm support for the lower back, abdomen, and pelvis, helping to alleviate pregnancy-related discomfort.
- **Compression:** Gentle and adjustable compression to relieve pressure on the lower back and improve posture without restricting movement or circulation.
- **Adjustability Features:** Features a fully adjustable hook-and-loop closure system to ensure a customized fit throughout the different stages of pregnancy. The belt includes an over-belly strap for added stability and support, which can be removed or repositioned as needed.
- **Materials Used:** Constructed with 80% polyester and 20% spandex, the maternity belt is safe, latex-free, and designed to be gentle on the skin and free of common allergens. It offers a lightweight, breathable, and stretchable design for maximum comfort and durability.

CONTRAINDICATIONS

⚠ Do not use if you have:

- Do not use the maternity belt if you have any open wounds, rashes, or skin infections in the areas where the belt makes contact with your skin.
- Not recommended for individuals with severe circulatory problems, such as deep vein thrombosis (DVT), unless advised by a healthcare provider.
- Avoid use if you experience any discomfort, excessive pressure, or pain while wearing the belt. If these symptoms occur, remove the belt and consult a healthcare professional.
- Not suitable for use immediately after certain medical conditions or surgeries without the approval of a healthcare provider.

APPLICATION INSTRUCTIONS

Putting On Your Support

1. **Position the Abdominal Support Pad:**

Place the pad under your belly and secure it by wrapping the strap around your body. Make sure it's low as possible without blocking your legs when you walk or sit.



2. Attach the Back Support Brace:

Connect one end of the brace to the abdominal pad, wrap it around your back, and secure it to the other side.



3. Secure the Elastic Tummy Strap:

Attach one end of the strap to the back brace. Stretch it lightly over your belly and secure it to the other side to keep the belt in place.



4. Adjust the Back Support:

Pull the elastic straps on each side of the back brace to customize the support and comfort.



PROPER FIT

- Belt should feel snug but not tight
- Support pad sits below abdomen
- Remove if you feel:
 - Discomfort
 - Restricted breathing
 - Pinching
 - Pain

USAGE GUIDELINES

- Wear up to 12 hours daily
- Take breaks every 4-6 hours
- Remove if uncomfortable

CARE INSTRUCTIONS

- Hand wash with mild soap
- Air dry flat
- No bleach or iron
- Store in cool, dry place

MANUFACTURE AND PRODUCT INFORMATION

Manufacturer



Vive Health

8955 Fontana Del Sol Way
Naples FL 34109

service@vivehealth.com
1-800-487-3808
vivehealth.com

EC	REP
----	-----

Compliance Management Solutions Europe

2 Bulgaria Str, 2850 Petrich,
Bulgaria.

SIGNS AND SYMBOLS

	<p>Medical device indication ISO/DIS 15223- 1:2020 Reference no. 5.7.7 Indicates the item is a Medical Device.</p>
	<p>CE marking Communauté Européenne Product traded on the extended Single Market in the European Economic Area (EEA); has been assessed to meet high safety, health, and environmental protection requirements.</p>
	<p>Consult instructions for use ISO 7000-1641 Indicates the need for the user to consult the Instructions for Use.</p>
	<p>Manufacturer ISO 7000-3082 To identify the manufacturer of the Medical Device.</p>
	<p>Date of manufacture ISO 7000-2497 To indicate the date on which the Medical Device was manufactured.</p>
	<p>Importer To identify the importer of the Medical Device.</p>
	<p>Caution ISO 15223-1, Clause 5.4.4 To indicate the need for the user to consult the Instructions for Use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the Medical Device itself.</p>
	<p>SUP3085</p>
	<p>Indicates a medical device that has not been subjected to a sterilization process.</p>
	<p>Latex Free</p>
	 <small>(01008101130791220241)SUP3085UKG</small>

IMPORTANT SAFETY NOTICE!!

If you experience any serious incident related to the use of this device, immediately discontinue use and report it to:

1. The manufacturer: Vive Health
(8955 Fontana Del Sol Way, Naples FL 34109)
2. Contact your national health authority or medical device regulator.
3. Your healthcare provider.

A serious incident is any incident that directly or indirectly led, might have led, or might lead to:

- Death of a patient, user, or other person.
- Temporary or permanent serious deterioration of a patient's, user's, or other person's state of health.
- A serious public health threat .

For any questions or concerns that are not serious incidents, please contact Vive Health customer service at.

service@vivehealth.com

1-800-487-3808

vivehealth.com

WARRANTY INFORMATION

Vive Health warrants this product to be free from manufacturing and material defects for a period of 90 days from the date of purchase under normal use. This warranty extends only to the original purchaser and does not cover:

- Damage from improper use, abuse, or modification
- Normal wear and tear
- Battery or replaceable components
- Products purchased from unauthorized sellers
- Commercial, institutional, or rental use

To obtain warranty service, contact Vive Health customer service with proof of purchase and a description of the defect. Vive Health will repair or replace defective products at our discretion.



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vhealth.link/rcm](https://vivehealth.com/rcm) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China