

WHEELCHAIR BAG ADVANCED

LVA3100



To see all the FAQ's,
visit [vhealth.link/fxv](https://vivehealth.com/faq)

vive
vivehealth.com

QUICK START GUIDE

COMPATIBLE WITH THE FOLLOWING:

- Wheelchairs
- Transport Chairs
- Power Wheelchairs

INTENDED USE

The Vive Wheelchair Bag advanced is designed as an accessory for mobility devices, specifically intended to provide secure storage capacity for personal belongings while using wheelchairs and transport chairs.

APPLICATION INSTRUCTIONS

Attaching the bag

1. Attach the buckles around the handle of the wheelchair.



2. Fasten the straps and ensure they are snug to the handle.



Using the Shoulder Strap

To attach shoulder strap, attach the hooks to the D-rings on either side of the bag.



QUICK START GUIDE

vive
vivehealth.com

CARE INSTRUCTIONS

DO'S

- Machine wash with cold water and mild detergent.
- Lay out flat to air dry.

DONT'S

- Do not bleach
- Do not machine dry
- Do not iron or dry clean

WARNINGS ⚠

- Do not use to carry medical devices that require secure positioning.
- This product is not intended to prevent, cure, or treat any medical condition.

WARRANTY INFORMATION

The Vive Wheelchair Bag is backed by a 1 year warranty from the date of purchase. This warranty covers defects in materials and workmanship under normal use.

If the product proves to be defective during the warranty period, we will, at our discretion, repair or replace the unit free of charge. Proof of purchase is required for warranty service.

Need Help? Contact Vive health:

Phone: 1-800-487-3808

Email: service@vivehealth.com

Website: vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com