

WHEELCHAIR TRAY

MOB1142



To see all of the FAQs, visit vivehealth.com/jfm

vive
vivehealth.com

QUICK START GUIDE

COMPATIBLE WITH THE FOLLOWING

- Standard Wheelchairs
- Transport Chairs

WHAT'S INCLUDED

- 1x Wheelchair Tray
- 2x Mounting Clamps
- 2x Mounting Screws
- 1x Non-Slip Mat
- 1x Quickstart

INTENDED USE

The Vive Wheelchair Tray is designed as an accessory for mobility devices, specifically intended to provide a stable, convenient surface for eating, writing, reading, or other daily activities while using a wheelchair.

APPLICATION INSTRUCTIONS

Attaching the Tray

1. Attach the clamps to the wheelchair armrest bars.



2. Secure each clamp using the included screws. Repeat on both sides.
3. Insert the support pegs into the holes on the underside of the tray. Use the inner holes for 16" wheelchairs and the outer holes for 18"-20" wheelchairs.
4. Slide the tray pegs into the clamps until fully seated.

Note: Depending on your wheelchair width, you may need to rotate the clamps inward or outward for the best fit.



QUICK START GUIDE

vive
vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com

5. Place the included non-slip mat on the tray for added stability.

Note: For optimal performance, position the non-slip mat with the shiny side facing down.



Removing the Tray

The tray can be lifted upward to remove, leaving the clamps in place for quick and easy reattachment.



CARE INSTRUCTIONS

DO's

- Wipe clean with a damp cloth and mild disinfectant.
- Dry thoroughly before reuse.

DON'Ts

- Do not submerge in water.
- Do not use harsh chemicals or abrasive cleaners.
- Do not place hot cookware directly on the tray.

WARNINGS

- Do not use as a transfer board or to support body weight.
- Do not use to carry medical devices that require secure positioning.
- This product is not intended to prevent, cure, or treat any medical condition.

WARRANTY INFORMATION

The Vive Wheelchair Tray is backed by a 6 month warranty from the date of purchase. This warranty covers defects in materials and workmanship under normal use.

If the product proves to be defective during the warranty period, we will, at our discretion, repair or replace the unit free of charge. Proof of purchase is required for warranty service.

NEED HELP? CONTACT VIVE HEALTH:

service@vivehealth.com

1-800-487-3808

vivehealth.com/support