


## HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vivehealth.link/6dd78](http://vivehealth.link/6dd78) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.

 [service@vivehealth.com](mailto:service@vivehealth.com)

 1-800-487-3808

 [vivehealth.com](http://vivehealth.com)

Distributed by

**vive**  
health

8955 Fontana Del Sol Way  
Naples, FL 34109  
Made in China

090922

vive®



## TRANSFER BELT

Owner's Manual  
RHB101IE

[vivehealth.com](http://vivehealth.com)



To see all of the FAQs,  
visit [vivehealth.link/6dd78](http://vivehealth.link/6dd78)

## QUICK START GUIDE

1. Insert strap into the buckle through the end with the teeth.



2. Slide strap through the hole opposite the teeth.



## QUICK RELEASE

To quickly release the strap, pull the release tab and slide the strap free.

