

# INSTRUCTIONS FOR USE

## VIVECARE Silicone Scar Tape

### DESCRIPTION


Silicone Scar Tape is a soft, thin, reusable, and self-adhesive silicone gel sheet mainly made from medical-grade silicone. It is effective for managing both existing and new hypertrophic or keloid scars and as a prophylactic treatment on closed wounds to prevent hypertrophic or keloid scarring.

### INDICATIONS


- For external use only.
- Temporary use in managing both existing and new hypertrophic or keloid scars.
- Temporary use as a prophylactic treatment on closed wounds to prevent hypertrophic or keloid scarring.

### INSTRUCTION FOR USE


1. Wash hands before use.

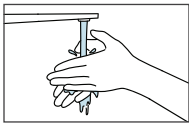


2. Clean and dry the scar area.

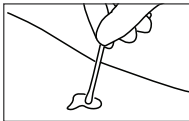


3. Cut the scar sheet.

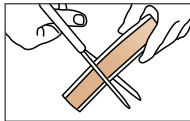

4. Peel off the liner.

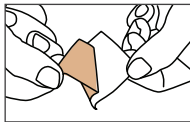

5. Apply the adhesive side directly onto the scar.

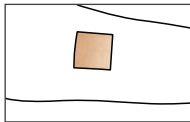







6. Wash silicone gel sheet, scar and surrounding skin daily with mild soap and rinse in clean warm water. The scar sheet can be reused for up to 7 days.

## WARNINGS

- DO NOT USE ON OPEN WOUNDS OR DAMAGED SKIN.
- Consult with a healthcare provider before using it on children.
- Consult a healthcare provider before using this product on facial scars, especially near the eyes.
- To reduce symptoms of discomfort and minor itching, recommend cleaning the scar area before use.
- Wash the product twice daily in a mild non-oily soap solution and rinse with clean warm water. After drying, the product can be reapplied to the scar site. Avoid using irritating detergents as they may lead to skin irritation. Paper products should not be used to dry the product as they can adhere to the gel sheet.
- When using the product in areas where sweating may occur, cleanse it more often for comfort and to maintain optimal contact between the product and the skin.
- The product should be worn for 4-8 hours per day initially, gradually increasing to 24 hours a day.
- Use for a minimum of 12 hours a day for maximum results.
- Avoid using ointments on the treated area as they may interfere with the product's performance.
- When the product begins to deteriorate or cleaning is difficult, replace it.
- The recommended duration of treatment is 2-4 months to achieve noticeable results.
- If a rash or allergic reaction occurs, discontinue use and consult your physician.

## SYMBOLS

	Manufacturer
	Caution
	Batch code
	Consult instructions for use
	Temperature limit
	Use-by date

## STORAGE

Store at room temperature, protected from sunlight.

**SHELF LIFE:** 5 years

**Distributed by:**  
VIVE HEALTH  
8955 Fontana  
Del Sol Way  
Naples, FL 34109  
1-800-487-3808  
Made in China