

INDUSTRIAL BACK BRACE

SUP3021BLK

INSTRUCTIONS

1. Place the back brace on the lower back and position the suspenders on the shoulder.

Assistance may be required



2. Use the hand and thumb loops to pull the straps to a comfortable fit then secure.



vive
vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com

3. Pull and attach the suspenders to the front belt, adjust to a comfortable fit.



4. Secure the two elastic straps to the desired level of compression.



WARNINGS

- Consult your doctor before use.
- Stop use if pain occurs or worsens.
- Assistance may be required during the application of this support.
- Do not over tighten, doing so may lead to restricted blood circulation.
- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, please consult your doctor immediately.
- Do not machine wash or dry the brace.

CARE

Hand wash with cold water. Hang dry.
Do not bleach. Do not machine wash or dry.